FOLLO DIEM G

FOLLO DIEM G

We congratulate you on choosing a Follo Diem product. This instruction enable you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.

Contents: Check list, unpacking/installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list

ORDER NO.:

SERIAL NO.:

FUNCTION TEST:

Bottom frame (welding, surface)

Surfaces (no damages)

Top frame (plastic parts, surface)

Adjustments(up/down)

ASSEMBLED (date/sign.):

END CONTROLLED (date/sign.):

PACKING CONTROL: (date/sign.):

FOLLO FUTURA AS - POSTBOKS 112 - 1430 ÅS - NORWAY

WWW.FOLLO-FUTURA.NO

EC DECLARATION OF CONFORMITY FOR MACHINERY AND MEDICAL DEVICES

Manufacturer: Follo Futura AS

Address: Brekkeveien 43
1430 Ås
NORWAY

Herewith declares that: Follo Diem G (147-223-00 and 147-223-01)

is in conformity with the provisions of the Medical Device Directive (93/42/EEC annex VII)

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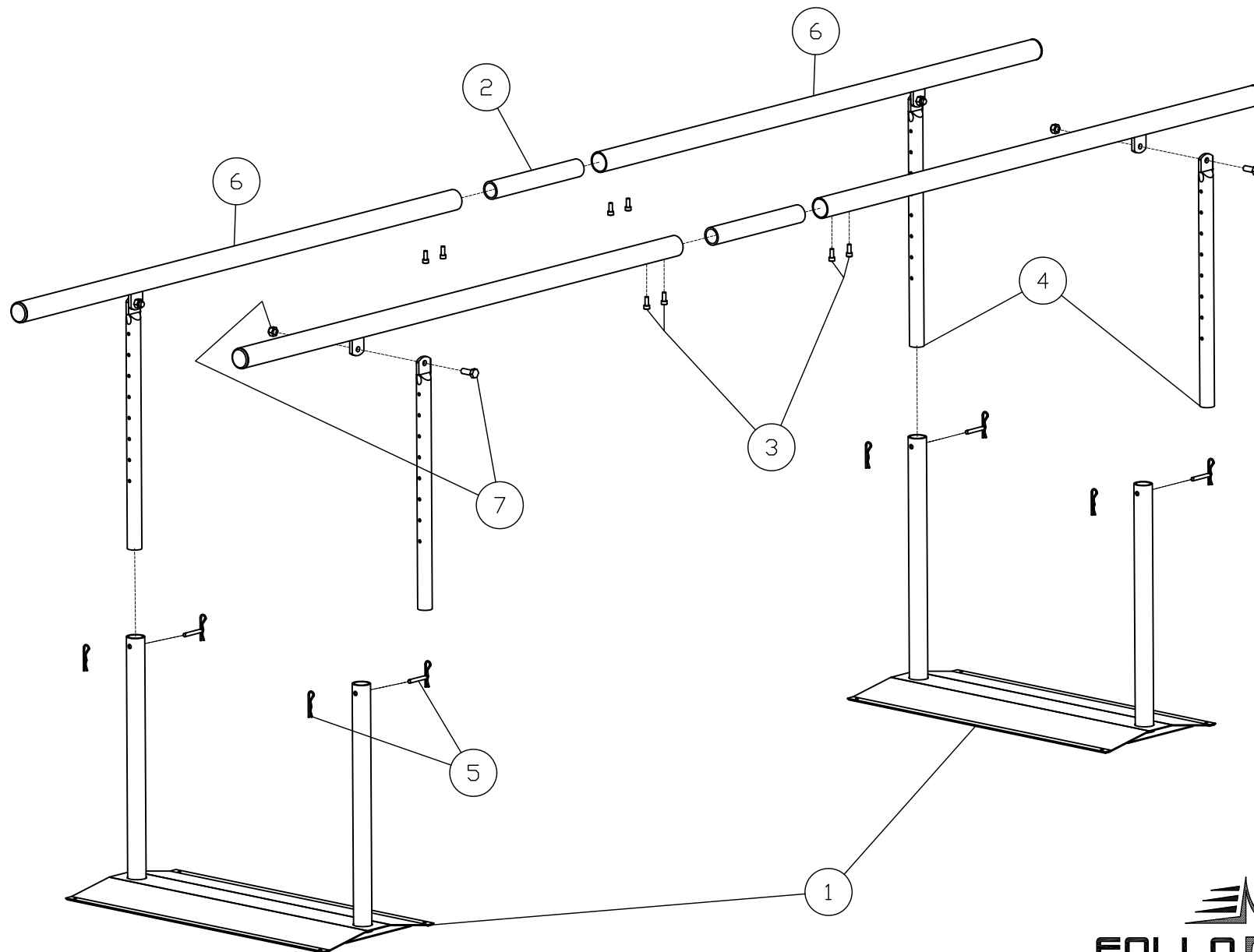
Unpacking / installation:

- Be careful when opening the packing. Knives or sharp objects must NOT be used because these can damage the product.
- Check to see whether the product was damaged during transport or not.

Safety information:

- Get to know the product.
- Make certain that the product is standing stable on the floor, anchored to the floor if necessary using the pre-drilled holes in the base plates.
- The parallel walking rails should only be used for walking training. Such training must only be done under the supervision of professional personnel.
- The walking rails must not be overloaded with more than the stated maximum weight; please see the technical information on the last page.

Guarantees:

The product must not be modified in any way or used with other than the original parts. Producer liability for CE-labelling is revoked if any modifications are made to the product.

- The product has a 5-year production guarantee on mechanical constructions.

This product has been mounted, checked and approved pursuant to the company's quality assurance system.

Guidelines for Use:

Assembly instructions, page 3:

- Place the floor plates (1) on the floor at approximate correct distances.
- Mount the joining sleeves (2) and fasten these properly with the screws that are included in the assembly package (3).
- Set the footing pipes (4) into the floor plates (1) and insert the latch pins (5).
- Connect the handrails (6) to the footing pipes (4) and screw this tightly into place using the screws and nuts that are included in the assembly package (7).
- The walking rail system may be anchored to the floor for better stability.

Adjustment settings:

Remove the cotter pins and latch pins (5) to adjust the height of the walking rail system. Adjust the walking rail system to the desired height, and put the latch pins and cotter pins back into the correct holes.

Maintenance / later inspections:

Check the following items once every three months:

- Make certain that the screws and nuts are still fastened securely.
- Make certain that the latch pins are not worn out.

Cleaning:

The product may be cleaned using ordinary disinfectants. **PLEASE NOTE!** Products with alcohol may damage the finish.

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