# FOLLO CORPUS G

 FOLLO FUTURA

EC DECLARATION OF CONFORMITY FOR  
MACHINERY AND MEDICAL DEVICES

---

Manufacturer: *Follo Industrier AS*

Address: *Brekkeveien 43*  
*1430 Ås*  
*NORWAY*

Herewith declares that: *FOLLO CORPUS G (124-XXX)*

- is in conformity with the provisions of the Machinery Directive (Directive 89/392/EEC, 91/368/EEC, 93/44/EEC annex I) and Medical Devices Directive (93/42/EEC annex VII)

Class I.

---

*19.11.02 Ås, Norway*

*For Mette Veiby: Developpe Manager Hans Kr. Södermann*

*Mette Veiby*  
*Managing Director*

# FOLLO CORPUS G

*We congratulate you on choosing a Follo Corpus product. This instruction information makes it able for you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.*

*Contents: Check list, unpacking/Installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list*

ORDER NO.:
COLOUR:
SERIAL NO.:

<b>FUNCTION TEST:</b>	<input type="checkbox"/> Cushions (colour, screws, surface)
<input type="checkbox"/> Bottom frame (screws, plastic parts, surface)	<input type="checkbox"/> Paper roll holder
<input type="checkbox"/> Top frame (screws, plastic parts, surface)	<input type="checkbox"/> 110 Voltage <input type="checkbox"/> 220 Voltage
<input type="checkbox"/> Surfaces (no damages)	<input type="checkbox"/> Labels (adjusting level, lock release, serial nr., warning signs)
<input type="checkbox"/> Rail (max/min)	<input type="checkbox"/> Tray/tray holder
<input type="checkbox"/> Raising (function/noise)	<input type="checkbox"/> Foot pedal, lowering/raising. (Screws)

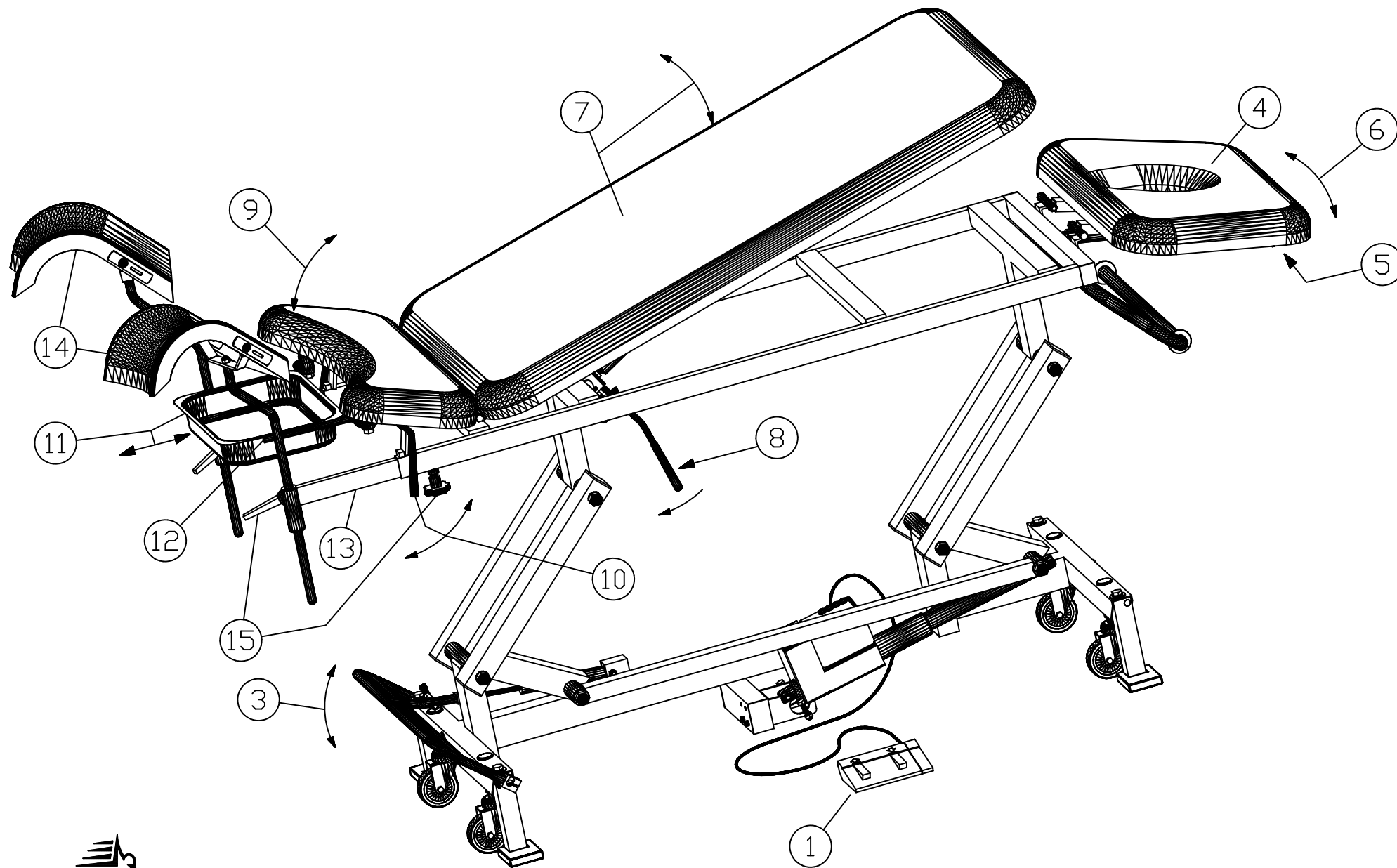
ASSEMBLED (date/sign.):	
-------------------------	--

END CONTROLLED (date/sign.):	
------------------------------	--

PACKING CONTROL: (date/sign.):	
--------------------------------	--

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# Follo Corpus G

## UNPACKING/INSTALLATION:

1. Take care when removing packing materials. DO NOT use a knife or other sharp instrument to avoid damaging the contents.
2. Inspect the bench for transport damage.

## SAFETY INSTRUCTIONS:

1. Study and learn the operation of the handles and switches.
2. Take care to avoid pinching when making adjustments.
3. Do not overload the bench.

## INSTRUCTIONS FOR USE:

1. Adjusting the height of electrically powered tables: → Use the pedal (1) to adjust to the required height.
2. Adjustment of head cushion (4): → The handle (5) in front of/below the head cushion releases the lock, enabling the cushion to be moved up or down (6). Releasing the handle locks the head cushion in position.
3. Adjusting the back cushion (7): → To raise or lower the back cushion, hold the cushion in one hand while operating the lever (8) with the other hand as shown in the drawing. The cushion is springloaded, pushing the cushion upwards. To lower the cushion, use one hand to push the cushion downwards, while holding the lever (8) with the other hand. Releasing the lever locks the back cushion into position. There are locking levers on both sides of the table.
4. Adjusting the seat cushion (9): → The seat cushion has two fixed positions: Horizontal or angled at 20 degrees (as shown in the drawing). To raise the seat cushion, hold the cushion in one hand and grip handle (10) with the other hand. Pull the cushion upwards and turn the handle backwards and downwards to the stop position (as shown in the drawing). Make sure that the handle stops against its stud on the frame below. To lower the seat cushion (9), hold the cushion in one hand and the handle (10) with the other hand. Lift the seat cushion up slightly and turn the handle forward to the stop position, lowering the seat cushion at the same time.
5. Stainless steel tray (11): → The stainless steel tray (11) should only be used when the seat cushion is in the raised position as shown in the drawing. To install the stainless steel tray (11), turn the arms of the tray holder (12) outwards so that they point backwards from the seat cushion (9). Insert the tray in position between the arms. The tray holder arms (12) are hinged so that they can be placed underneath the seat cushion when the tray is not in use.
6. Adjusting the leg rests (14): → The supports (13) for the leg rests are supplied with the table as standard equipment, but the leg rests (14) are extras. The leg rests can be adjusted to the desired position, using the two locking wheels (15) on either side of the table.

## MAINTENANCE INSTRUCTIONS:

Check the following at regular intervals (every third month):

1. Make sure that all wheels run smoothly.
2. Make sure that the power cables are not damaged or chafed.
3. Check that the height adjustment and that all handles, levers and pedals are functioning properly.
4. Wash and dry the bench and apply a thin layer of ordinary silicon spray to all sliding surfaces.

## Cleaning:

To maintain the appearance of PVC Coated Fabrics they should be cleaned regularly to remove fatty substances, which may reduce their service life. A soft cloth and warm soapy water should be used to clean the surface of the material. Mild soap or soap flakes are recommended and residual soap should be removed using cold water. The surface should be scrubbed with a soft brush. Solvent cleaners, wax polishes strong detergents and patent chemical cleaners should not be used. The use of these substances is likely to be harmful to the Coat and repeated use can result in the removal of elasticity from the PVC coating, which will result in hardening and subsequent cracking of the material's surface. PVC Coated Fabrics are resistant to most mild acids, alkalis and household stains. Some substances such as ballpoint pen ink, lipstick, newsprint and foodstains may be absorbed by the vinyl and cause permanent staining. This can often be minimised by immediate cleaning with damp, soap cloth or sponge. Never place hot or cold items directly on PVC Coated Fabrics.

Hydraulic models: Hydraulic tables are equipped with a long-life heavy hydraulic lifting unit. However, the unit some times after transport needs an airing. To do that, activate the foot-pedal 4-6 times after the table has come up to maximum level.

Electric models: Electrically powered tables are equipped with a long-life maintenance-free electric motor. Should any problems occur with this unit please check the following items:

1. Check the plug to see that it is securely plugged into the electric outlet.
2. Check the electric plug on the electric motor to see that it is fastened properly.

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