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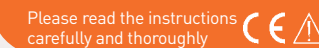
REF. P35A PLUS
3-point hyperextension brace
acqua

Please read the instructions carefully and thoroughly

Headquarter: ORTHOSERVICE AG
Via Milano 7 - CH-6830 Chiasso (TI) - Switzerland
info@orthoservice.com - www.orthoservice.com
Niederlassung Deutschland: Orthoservice Deutschland GmbH
Flugstraße 7 - D-76532 Baden-Baden - Deutschland
info@orthoservice.de - www.orthoservice.de
Sede italiana: ROTEN s.r.l.
Sede legale: Via Fratelli Ruffini, 10 - I-20123 Milano (MI) - Italia
Sede operativa e amministrativa:
Via Comasina, 111 - I-20843 Verano Brianza (MB) - Italia
info@roplusten.com - www.roplusten.com
Società soggetta a Direzione e Coordinamento (art. 247bis CC):
Orthoservice AG (CH) - 6830 Chiasso (TI) - Switzerland

Take care feel better
ORTHOSERVICE ROTEN

SCOPE OF DELIVERY

Hyperextension brace Modular plus includes:

- A** 1 x Frame made from light aluminium alloy with safety lock closure
- REF. P35K PLUS - REF. P35A PLUS - REF. P36 PLUS - REF. P45 PLUS**
- B** 1 x Fluctuating sternal plate
- 1 x Back plate (lumbar plate) (without fig. I)
- REF. P38K PLUS**
- C** 1x Delta-pectoralis pads fastened on crescent shaped bar
- REF. P36 PLUS**
- D** 1 x Fixed pubic pad
- REF. P35K PLUS - REF. P35A PLUS - REF. P38K PLUS - REF. P45 PLUS**
- E** 1 x Floating pelvic band with spring (can be blocked)
- REF. P45 PLUS**
- F** Toothed wheel under the arm to adjust the pressure against the sternum
- REF. P35A PLUS (acqua)**
- Padded parts and belt in water-repellent material, suitable for use in water

CHARACTERISTICS AND MATERIALS

- Frame made from light aluminium alloy
- Frame construction
- Back pad can be positioned either horizontally or vertically
- Adjusted via screws
- Lever closure made of plastic with safety lock
- Newly designed strap with special hook closure system for adjusting the length
- Underarm protective lining in non-toxic, non-allergic material
- Adjustable Velcro® straps
- **REF. P35K PLUS - REF. P35A PLUS - REF. P45 PLUS:** Movable sternal plate
- **REF. P38K PLUS:** Delta-pectoralis pads mounted to a half-moon plate as replacement for the sternal pad
- **REF. P35K PLUS - REF. P35A PLUS - REF. P45 PLUS REF. P38K PLUS:** Spring oscillation of the pelvic strap (with the option of securing at 0° and 18°)
- **REF. P36 PLUS:** Fixed pubic pad
- **REF. P35A PLUS:** Padded parts and belt in water-repellent material, suitable for use in water
- **REF. P45 PLUS:** Toothed wheel under the arm to adjust the pressure against the sternum: each tooth equals 5° setting variation
- **REF. P35K PLUS - REF. P36 PLUS - REF. P45 PLUS REF. P38K PLUS:** Padded parts made of soft foam with non-toxic, synthetic leather cover

INDICATIONS

- Traumatic fractures (T10-L1/L2)
- Osteoporotic or metastatic-related compression fractures of the thoracic spine and the dorsolumbar transition (T10-L1/L2)
- Osteomalacia with fractures
- Dorsolumbar osteoarthritis
- Substitute for plaster corset

ADAPTING FOR THE DOCTOR / TECHNICIAN

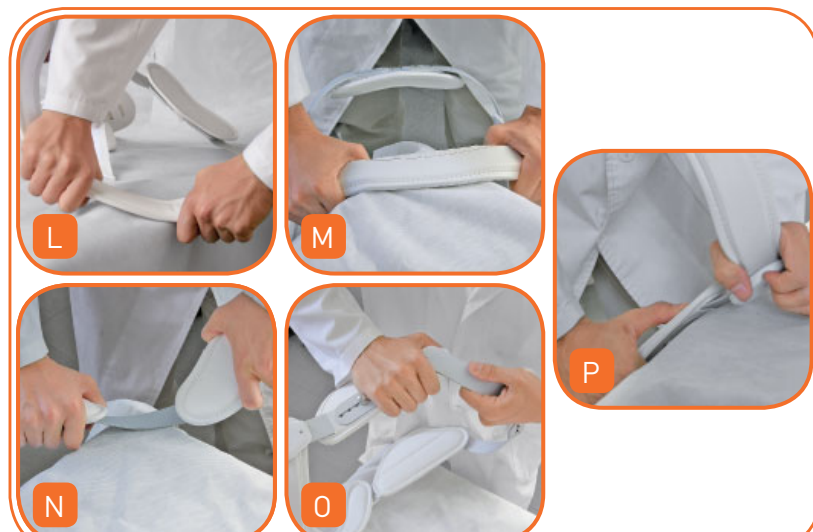
REF. P35K PLUS · REF. P35A PLUS

- 1** Apply the hyperextension brace frontally, with the patient lying down (fig. A).
- 2** Loosen the screws on the lateral bars (fig. B) on the front (fig. C).
- 3** Proceed to make the adjustments on the vertical axis (fig. D):
 - lengthen or shorten the hyperextension brace until the pelvic band rests on the pubis and the plate on the sternum; if necessary, remove the screws and place them in a more suitable position to adapt the hyperextension brace to the patient
 - tighten the screws and make sure the patient can sit without discomfort; if necessary, make further vertical adjustments
 - make sure the lateral bars are the same length after adjustment
- 4** Now make the horizontal adjustments (fig. E):
 - adapt the hyperextension brace to the width of the patient's chest
 - tighten the screws, checking the symmetry with respect to the sagittal plane
- 5** Adjust the brace accurately to adapt it to the morphology of the patient:
 - shape the pelvic band (fig. L and M)
 - shape the armpit sections (fig. N and O), so that they adhere well to the pectoral area
 - shape the lateral bars at the level of the pelvis (fig. P), following the reference fold (to shape them more accurately we recommend using bending tools)

REF. P38K PLUS

- 1** Apply the hyperextension brace frontally, with the patient lying down (fig. A).
- 2** Loosen the screws on the lateral bars (fig. B) on the front (fig. C).
- 3** Proceed to make the adjustments on the vertical axis (fig. D):
 - lengthen or shorten the hyperextension brace until the pelvic band rests on the pubis and the plate on the sternum; if necessary, remove the screws and replace them in a more suitable position to adapt the hyperextension brace to the patient
 - tighten the screws and make sure the patient can sit without discomfort; if necessary, make further vertical adjustments
 - make sure the lateral bars are the same length after adjustment
- 4** Now make the horizontal adjustments (fig. E):
 - adapt the upper part of the hyperextension brace to the width of the patient's chest
 - tighten the screws, checking the symmetry with respect to the sagittal plane
- 5** Adjust the delta-pectoralis pads so as to obtain a proper support:
 - tighten the screws all the way to lock them in this position (fig. F)
 - shape the connecting strip between the pads to fit the patient's morphology (fig. G)
- 6** Adjust the brace accurately to fit the morphology of the patient:
 - shape the pelvic band (fig. L and M)
 - shape the armpit sections (fig. N and O), so that they adhere well to the pectoral area
 - shape the lateral bars at the level of the pelvis (fig. P), following the reference fold (to shape them more accurately we recommend using bending tools)

ADAPTING FOR THE DOCTOR / TECHNICIAN

REF. P36 PLUS

- 1** Apply the hyperextension brace frontally, with the patient lying down (fig. A).
- 2** Loosen the screws on the lateral bars (fig. B) on the front (fig. C) and on the pubic pad (fig. K).
- 3** Proceed to make the adjustments on the vertical axis (fig. D):
 - lengthen or shorten the hyperextension brace until the pubic pad rests on the pubis and the plate on the sternum; if necessary, remove the screws and place them in a more suitable position to adapt the hyperextension brace to the patient
 - tighten the screws and make sure the patient can sit without discomfort; if necessary, make further vertical adjustments
 - make sure the lateral bars are the same length after adjustment
- 4** Now make the horizontal adjustments (fig. E):
 - adapt the upper part of the hyperextension brace to the width of the patient's chest; tighten the screws, checking the symmetry with respect to the sagittal plane
 - adjust the lower part to the patient's waist width, tighten the screws checking the symmetry with respect to the sagittal plane
- 5** Adjust the brace accurately to fit the morphology of the patient:
 - shape the bars of the pubic pad
 - shape the armpit sections (fig. N and O), so that they adhere well to the pectoral area

REF. P45 PLUS

- 1** Apply the hyperextension brace frontally, with the patient lying down (fig. A).
- 2** Loosen the screws on the lateral bars (fig. B) on the front (fig. C).
- 3** Proceed to make the adjustments on the vertical axis (fig. D):
 - lengthen or shorten the hyperextension brace until the pelvic band rests on the pubis and the plate on the sternum; if necessary, remove the screws and place them in a more suitable position to adapt the hyperextension brace to the patient
 - tighten the screws and make sure the patient can sit without discomfort; if necessary, make further vertical adjustments
 - make sure the lateral bars are the same length after adjustment
- 4** Adjust the inclination of the underarm section:
 - Remove the screw and metal cap from the hinge between the vertical upright and the underarm section (fig. H)
 - Raise the underarm section, turn to the desired inclination and fit it back on the toothed wheel (fig. I).
 - Fit the metal cover and screw back in place, and tighten the screw
 - Repeat the same operation on the other underarm section, taking care to set the same inclination on both sides.
- 5** Now make the horizontal adjustments (fig. E):
 - adapt the hyperextension brace to the width of the patient's chest; tighten the screws, checking the symmetry with respect to the sagittal plane
- 6** Adjust the brace accurately to adapt it to the morphology of the patient:
 - shape the pelvic band (fig. L and M)
 - shape the armpit sections (fig. N and O), so that they adhere well to the pectoral area
 - shape the lateral bars at the level of the pelvis (fig. P), following the reference fold (to shape them more accurately we recommend using bending tools)

POSITIONING OF THE LUMBAR PAD (FOR ALL MODELS)

- 7** In accordance with the physician's instructions, position the lumbar plate along the spinal column in vertical or horizontal position where prescribed (fig. R):
 - remove the knob screw and place it at the desired height (in one of the special brass bushings) and tighten it
 - move the lever to the desired height using the two screws that fasten it to the metal structure (fig. S)
 - in case of vertical positioning of the lumbar pad, remove the strap from the horizontal buckles and thread it through the vertical ones
- 8** Adjust the belt length:
 - having the closure lever open slide the belt into the buckles and fasten it with the corresponding Velcro® fastener
 - fit the hook in the knob screw
 - if necessary, make further adjustment to the straps to obtain the desired pressure in the lumbar area: the optimum pressure is obtained when the hyperextension brace remains in place even with the closure lever open (fig. T)
- 9** Fasten the hyperextension brace using the closure lever (fig. T).
- 10** Make sure the lumbar pad is centered securely on the spinal column and secure it using the strap and the corresponding buckles (fig. U).

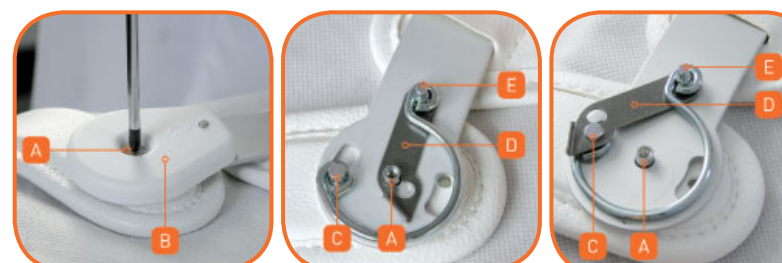
INSTRUCTIONS FOR BLOCKING THE PELVIC BAND (for the physician or orthopedic expert)

If necessary, it is possible to block the moving mechanism of the pelvic band (pre-set on delivery) by adjusting the lateral spring mechanisms. Follow the following procedure:

- I.** Loosen the central screw (A). Remove the plastic cover (B).
- II.** Move the pelvic band with one hand, while moving the block plate (D) and mounting it on the outer pin (C) with the other hand. (The pelvic band can be blocked in 2 positions)
- III.** Reattach the plastic cover by mounting it on the fixation pin (E) first, and on the central screw thread afterwards. Tighten the screw.
- IV.** Proceed in the same way on the opposite side.

Make sure that the mechanisms are blocked in the same manner, with the block plate (D) in the same position.

PUTTING ON THE APPLIANCE FOR THE PATIENT

- 1** Open the closure lever and unhook the belt from the knob screw (A).
- 2** Position the hyperextension brace on the chest, pass the belt behind the back and fasten it to the knob screw (A).
- 3** Position the hyperextension brace so that the sternal plate is centered on the chest (B) and the pelvic band is not uncomfortable when sitting (C).
- 4** Fasten the hyperextension brace using the closure lever (D).

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares with sole responsibility that this product is a Class I medical product and has been manufactured in accordance with the basic requirements of the directives 93/42/EEC and 2007/47/EEC. These operating instructions were created in accordance with the specification of the above-mentioned directives. They aim to ensure the safe and correct use of the medical product.

MAINTENANCE

Hand wash at max. 30° C with neutral soap (a sponge is recommended); do not dry in the vicinity of heat sources (or, better, dry with a cloth) Check screws for tight fit regularly. Contact an orthopaedic technician for the replacement of used parts. Do not dispose of the product or any of its components into the environment.

PRECAUTIONS

It is advised that the pressure exerted by the product does not act on parts of the body suffering from wounds, swelling or lumps. The product should not be tightened too firmly so as to avoid the development of pressure points or compression of the nerves or blood vessels underneath. It is advisable to wear the product over a garment, avoiding direct contact with the skin. If in doubt about how to apply, contact an orthopaedic technician. The device should not be worn in the vicinity of open flames or strong electromagnetic fields. Do not release the tension of the belt abruptly, in order to prevent sudden collapse of the trunk.

WARNINGS

The product may only be used in the event of medical indication or when prescribed by a physician and must be applied by an orthopaedic technician, the competent authority figure for the application as well as for information on the safe use, according to individual needs. To ensure the effectiveness, tolerability, and correct operation, it is necessary that the device is put on with the utmost care. Any changes to the structure or adjustment of the device must be prescribed by a physician and performed by an orthopaedic technician. Under no circumstances should the adjustment made by the physician/orthopaedic technician be altered. Use of the product is recommended for a single patient. With hypersensitive patients, direct contact with skin may cause redness or irritation. In the case of onset of pain, swelling, lumps or any unusual reaction, immediately contact your physician.

TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B. V.

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Sede italiana: ROT-TECH S.r.l.
Orthoservice AG (CH - 6830 Chiasso (TI) - Switzerland