

MOBIDERM® AUTOFIT




→ Product key features

- **Mobiderm® armsleeve along with mitten and thumb** : 80% of upper limb garments sales
- **Self-adjustable garment** : can be adapted to the individual morphology of the patient... and its long term evolution
 - 7 velcro tabs along the sleeve: **strong holding** and **easy positioning**
 - Printed tightening levels (+ / ++ / +++) at 4 key points (hand, wrist, forearm, axillary) **to easy adjust the product, and reproduce the fitting**



→ Product key features

- **Open conception** of the product: **easy to slip on, easy to remove!**
- **Mix of big blocks** along the arm / **small blocks** on the dorsal part of the hand : **best efficiency/tolerance ratio and best efficacy on the hand oedema**
- **Plain sleeve along with a mitten** : **in line with medical recommendations** ... but hand part easy to roll up to enable manual activities

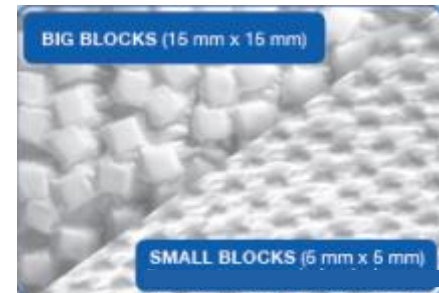

MOBIDERM® AUTOFIT

→ Elegant design

- **Tone on tone finish** (like standard garments):
elegant with discretion
 - Soft external material : 59% polyamide / 41% elastane
 - Quilted velvety on the fastening area for a better holding of internal Mobiderm material
 - Leather-look tabs



→ A serie product :

- **Quicker delivery time** to patients!
- **A wide range of sizes** to offer best patient coverage ...



→ Size range


- **6 different sizes**, according to **4 key measures (circumferences)** :
 - axillary,
 - Forearm,
 - wrist,
 - hand
- **2 arm lengths** (normal – long)
- **2 sides** (left/right)



→ Size range

	Tour de bras axillaire Axillary level (E)	Tour de l'avant bras Forearm level (D)	Tour de poignet Wrist level (C)	Tour de main Hand level (B)
T1	25 - 32 cm	25 - 31 cm	15 - 19 cm	17 - 21 cm
T2	29 - 36 cm	27 - 33 cm	16 - 20 cm	17 - 21 cm
T3	33 - 40 cm	29 - 35 cm	17 - 21 cm	18 - 22 cm
T4	37 - 44 cm	31 - 37 cm	17 - 21 cm	18 - 22 cm
T5	41 - 48 cm	33 - 39 cm	18 - 22 cm	19 - 23 cm
T6	45 - 52 cm	34 - 40 cm	19 - 23 cm	20 - 24 cm

Longueur de bras - Arm length (c)	
Normal	40 - 45 cm
Long	45 - 50 cm

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

Target sizes
 -> **The garment must be tried on the patient**

→ How to adjust the product?



A first indication

for a **standard fitting** (relying on 15mmHg pressure level)

is given by Thuasne (part of user manual)



Instructions to position the 4 major hooks

(hand, wrist, forearm and axillary) according to

- the size of the garment delivered
- and the patient's individual measures

Then position the intermediate hooks and the hand accordingly



This could secondly be adapted by the patient according to his feeling and comfort.



→ How to adjust the velcro tabs?

Axillary level						
Axillary circumf.	T1	T2	T3	T4	T5	T6
25	+++					
26	+++					
27	++					
28	++					
29		+++				
30	+	+++				
31	+	++				
32		++				
33			+++			
34		+	+++			
35		+	++			
36			++			
37				+++		
38			+	+++		
39				++		
40				++		
41					+++	
42				+	+++	
43					++	
44					++	
45						+++
46					+	+++
47						++
48						++
49						
50						
51						+
52						



Example :
 A patient with a Mobiderm Autofit size 3 and axillary circumference of 36 cm should position the axillary hook in the « ++ » area

→ Washing instructions

2 options are now possible :

- **Hand wash:** soak the system in cold soapy water, rub with a clean sponge and squeeze out excess water.
- **Machine wash:** use the wool programme, cold temperature and place in a wash bag. Squeeze out excess water by hand.


In both cases, allow to dry without using direct heat.

When the sleeve is not being worn, it should preferably be stored flat, with the velcro tabs fastened.





THUASNE
SINCE 1847






	Tour de bras cubitale (E)	Tour de l'avant bras l'horizontale (D)	Tour de poignet Wrist level (C)	Tour de main Hand level (B)
T1	25 - 32 cm	25 - 31 cm	15 - 19 cm	17 - 21 cm
T2	29 - 36 cm	27 - 33 cm	16 - 20 cm	17 - 21 cm
T3	33 - 40 cm	29 - 35 cm	17 - 21 cm	18 - 22 cm
T4	37 - 44 cm	31 - 37 cm	17 - 21 cm	18 - 22 cm
T5	41 - 48 cm	33 - 39 cm	18 - 22 cm	19 - 23 cm
T6	45 - 52 cm	34 - 40 cm	19 - 23 cm	20 - 24 cm


Longueur de bras - Arm length (A)

Normal	40 - 45 cm
Long	45 - 50 cm


THUASNE

THUASNE SAS SA, rue Marthe Audin - CS 10 032 - 92309 Levallois-Perret Cedex - France
Commandes/Orders (St-Germain) : Tél. 04 77 81 40 42 (France) - Tél. +33 (0)4 77 81 40 01/02 (Dopo 9)
www.thuasn.com








986_417401 | Site de fabrication de produits : 98610033