

Gymna ShockMaster

More than just shockwave therapy



gymna[®]

gymna.com

Gymna ShockMaster


Maximum results in a minimum of time

Radial shockwave therapy

The ideal treatment method for chronic and complex musculoskeletal conditions.

Operating principle

A rapidly-moving projectile transfers its energy to the treatment head. The radial shockwaves that are generated spread out and are absorbed by the area of the body that has to be treated.

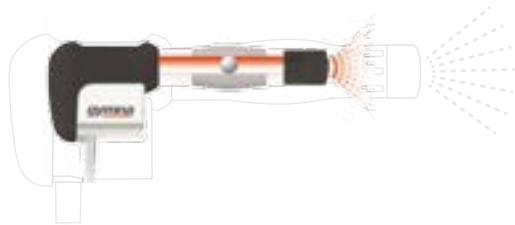

Results

- Pain reduction
- Regeneration of affected tissue
- Breakdown of calcification and fibrosis

Added value for your practice

- 6 to 8 treatment sessions required on average
- A realistic alternative to surgery



Gymna ShockMaster

Principle indications

Efficacy of Gymna ShockMaster in the treatment of:



Chronic tendinopathy and enthesopathy: epicondylalgia, plantar fasciopathy, patellar tendinopathy, rotator cuff tendinopathy (with or without calcification), etc.



Sports indications: proximal tendinopathy of hamstring muscles, adductor tendinopathy and peroneal tendinopathy, medial tibial stress syndrome, Jumper's knee, Osgood-Schlatter disease, etc.



Myofascial pain syndrome: lumbago, cervicalgia, trigger points, tension headache, etc.



Muscular affections: muscular pain, hypertonia, spasms, cramps, muscular contracture, fibrosis, etc.

Also: Dupuytren's contracture, Ledderhose disease, acupuncture points, metatarsalgia.

Gymna ShockMaster 500

Treat under the best possible conditions


The ideal support for your practice with a superior interface

User friendly

- Two output channels for simple and rapid switch between hand pieces
- Noiseless and vibration-free oil compressor, easy accessible
- Soft-close drawer with shock absorbers
- Rapid software update via USB port



Ease of movement with large wheels

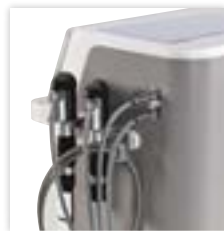

Practical cable holder system for hand piece



Well-organised accessories thanks to functional storage tray

Always up to date

Shockwave therapy is evolving constantly. Thanks to treatment software updates, you stay abreast of new developments:

- Treatment parameters are adapted in function of the latest information
- New indications are added, as well as relevant treatment protocols

Additional functions are:

- Real treatment photos in support
- Anatomical library
- Medical information on indications treated by shockwaves

All this provided in the most intuitive and effective way



Updated list of indications in function of therapeutic expertise and evidence-based studies



Patient database with treatment evolution on basis of VAS



Visual indication of the treatment zone with related pain during treatment of trigger points

Gymna ShockMaster 300

Radial shockwave therapy – anytime, anywhere

A compact format for more mobility



Mobile

- Small and light
- Easy to transport thanks to the handy carrier bag
- Silent built-in air compressor, easy to service

Simple

- Plug & Play
- Large control buttons
- Clear representation of treatment time thanks to coloured luminous bar



Handle the device while in its carrier bag



Gymna ShockMaster

Shockwave treatment adapted to the patient and pathology

1

Treatment of the affected area with the correct applicator



2

D-ACTOR :

- Relaxation of muscular tension
- Improvement of blood circulation
- Treatment of trigger points










3

V-ACTOR

- Stimulation of metabolic activity and elimination of toxins
- Muscle relaxation
- Improvement of microcirculation



15 mm Applicator		Tendinopathies and calcification up to 40mm deep
15 mm Focus lens applicator		Painful superficial areas, facial muscles, neck and head
6 mm Acupuncture applicator		Acupressure
15 mm Deep Impact applicator		Tendinopathies and calcification up to 60mm deep
20 mm D-ACTOR I		Myofascial pain therapy, fibrosis, muscle- and connective tissue, treatment of deep muscles (m. piriformis, m. gluteus minimus).
35 mm D-ACTOR II		Myofascial pain therapy, fibrosis, muscle- and connective tissue, back muscles, skin elasticity, treatment of deep muscles (m. piriformis, m. gluteus minimus).
V-ACTOR		Myofascial therapy, delayed onset muscle soreness, muscular treatment before and after physical effort.



Fast and easy servicing

Gymna ShockMaster

The best support and addition for your practice

Gymna ShockMaster means support, expertise and user-friendliness.

Advantages for the therapist include:



Knowledge and expertise

- Updating of the equipment with the latest treatment software based on expertise and evidence-based studies



Broader application

- New treatments constantly added
- New in 2014 – indications for sports
- Faster results thanks to the additional treatment possibilities




More patients with faster results

- Shockwave therapy possible for a greater number of patients
- Your patients will regain their activity level more rapidly
- An incentive to develop your practice



An investment that offers maximum return

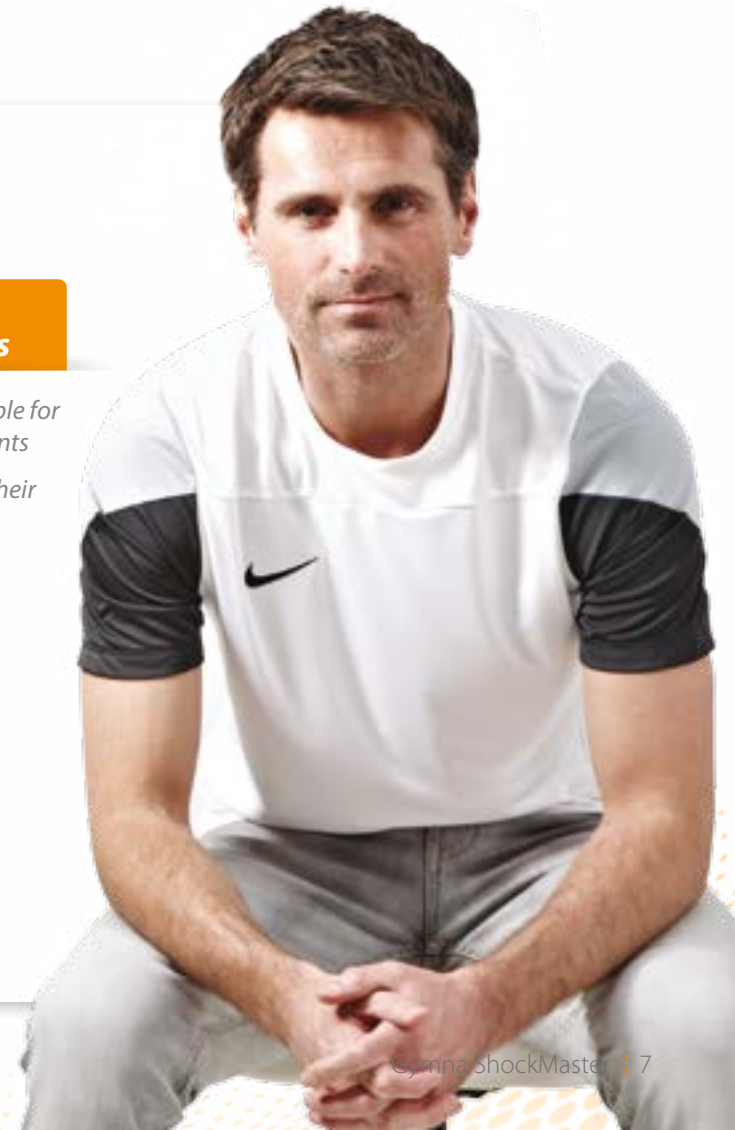

NEW

Gymna Physio Care

Top quality massage lotions for professionals



Cryotherapy



Diathermy



Shockwave Therapy



Physio Care



Treatment Couches



Electrotherapy



Active Motion



Your dealer

gymna.com

gymna[®]

GymnaUniphy NV Pasweg 6A | B-3740 Bilzen, Belgium

Tel: +32 (0)89 510 510 | Fax: +32 (0)89 510 511