# FOLLO DIEM 3-SECTION

 FOLLO FUTURA

# FOLLO DIEM 3-SECTION

*We congratulate you on choosing a Follo Diem product. This instruction information makes it able for you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.*

*Contents: Check list, unpacking/Installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list*

ORDER NO.:

COLOUR:

SERIAL NO.:

## FUNCTION TEST:

Frame (screws, plastic parts, surface, welding)

Labels

Cushions (colour, screws, surface)

Adjustments

ASSEMBLED (date/sign.):

END CONTROLLED (date/sign.):

PACKING CONTROL: (date/sign.):

**FOLLO FUTURA AS - POSTBOKS 112 - 1430 ÅS - NORWAY**

**WWW.FOLLO-FUTURA.NO**

---

## EC DECLARATION OF CONFORMITY FOR MEDICAL DEVICES

---

*Manufacturer:* Follo Futura AS

*Address:* Brekkeveien 43  
1430 Ås  
NORWAY

*Herewith declares that:* Follo Diem 3-section bench (136-012902)

and extra equipment:

Follo Diem head cushion (136-012902-01)  
Follo Diem foot cushion (136-028930)  
Follo Diem foot cushion top for 3-section (136-028931)  
Follo Diem shoulder rotating trainer (136-028936)

is in conformity with the provisions of Medical Device Directive (93/42/EEC annex VII)

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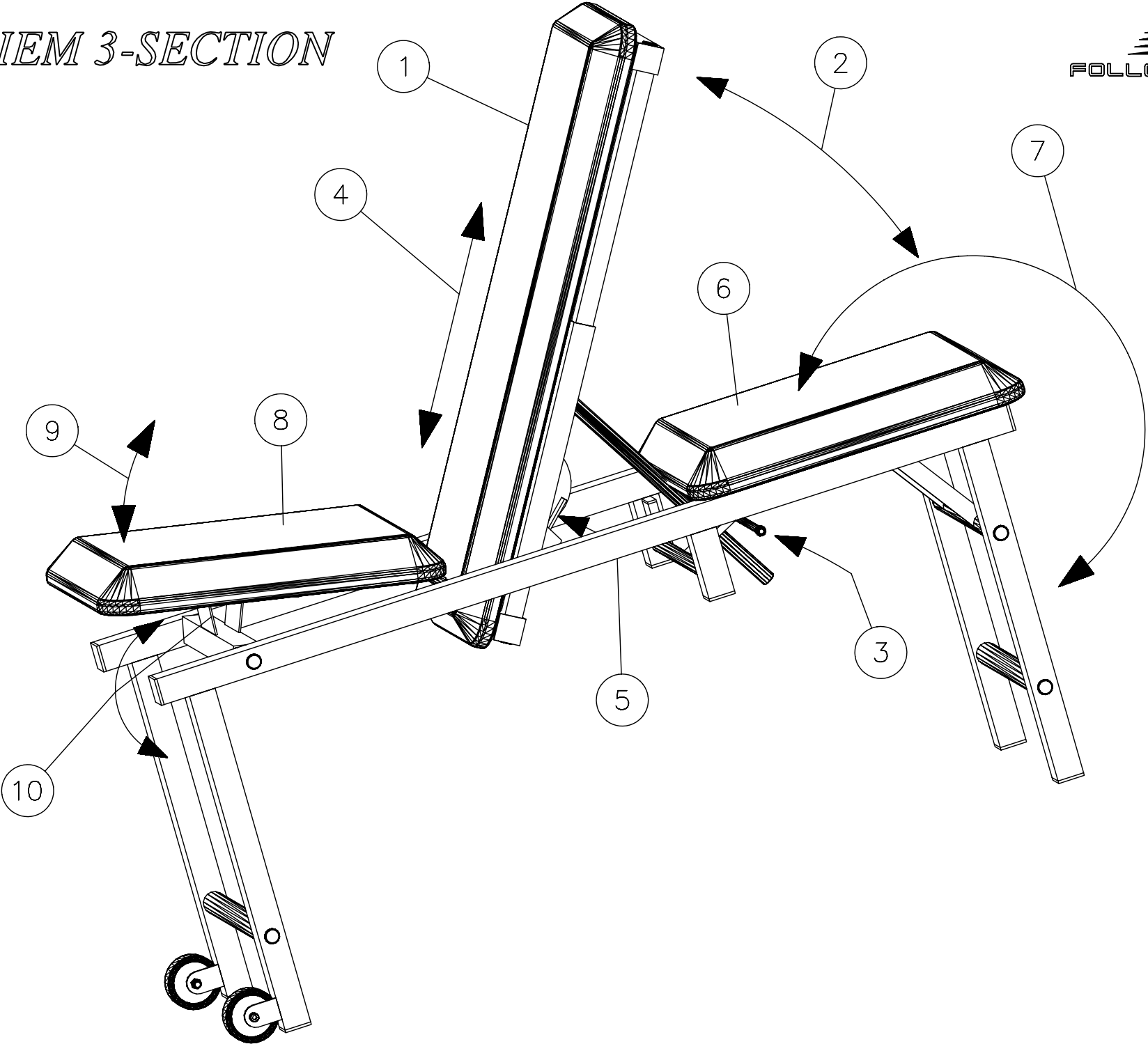
# FOLLO DIEM 3-SECTION

We congratulate you on choosing a Follo Corpus product. This manual enables you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use. Contents:

- Unpacking/installation
- How to use
- Technical specifications
- Safety information
- Maintenance instructions
- Spare parts list

**NB! The manufacturer responsibility for CE marking is invalidated if any form of modification is carried out.**

## Unpacking/installation:

Take care when removing packing materials. **DO NOT** use a knife or other sharp instrument to avoid damaging the contents.

Inspect the bench for transport damage.

Study and acquaint yourself with the adjustment options. All bench sections are adjustable.

## Safety instructions:

When adjusting the sections, take care to avoid pinching.

Do not exceed the specified weight limit (120kg).

## How to use:

1. Angle adjustment of back cushion (1): → Grip the back cushion and lift up to the required angle (2). Operate the release arm (3) to lower the cushion. Hold the cushion (3) and adjust to the required angle while holding up the release arm.
2. Height adjustment of back cushion (1): → When the back cushion is in the raised position, it can also be adjusted for height (see illustration). The release arm (5) underneath the back cushion releases the locking device enabling in position. Make sure the release arm is properly engaged.
3. Adjustment of rear cushion (6): → The rear cushion can be placed in two positions; horizontally on the top of the bench or vertically at the end of the bench (see illustration). Adjust the cushion by gripping it and tilting it up or down (7). There is no locking device on the rear cushion as it is immovable when it is in position.
4. Adjustment of seat (8): → The seat has two-position adjustment: horizontal or angled at approx. 10° (see illustration). To adjust the seat, grip it and tilt up (9), and then rotate the supporting tube underneath the seat (10) upwards or downwards and place the seat so that it rests on the supporting tube.

## Maintenance instructions:

Check the following at regular intervals (every third month):

Check that the cushions are properly fixed in position and the hinging and locking devices function properly.

Check that the cushions are not damaged or worn.

Check that the bench is stable.

To maintain the appearance of PVC Coated Fabrics they should be cleaned regularly to remove fatty substances that may reduce their service life. A soft cloth and warm soapy water should be used to clean the surface of the material. Mild soap or soap flakes are recommended and residual soap should be removed using cold water. The surface should be scrubbed with a soft brush. Solvent cleaners, wax polishes strong detergents and patent chemical cleaners should not be used. The use of these substances is likely to harmful the coat and repeated use can result in the removal of elasticity from the PVC coating, which will result in hardening and subsequent cracking of the material's surface. PVC Coated Fabrics are resistant to most mild acids, alkalis and household stains. The vinyl may absorb some substances such as ballpoint pen ink, lipstick, newsprint and food, and this may cause permanent staining. This can often be minimised by immediate cleaning with damp, soap cloth or sponge. Never place hot or cold items directly on PVC Coated Fabrics.

## Technical specifications:

Height:	47cm
Width:	32cm
Length:	100cm
Weight:	19kg

## FOLLO DIEM HEAD CUSHION

When ordering, please quote No. 136-012902-01

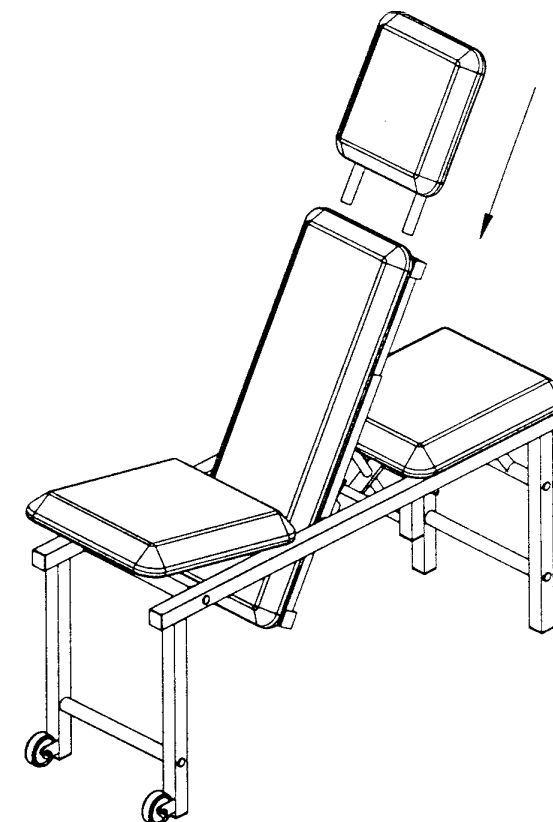
*A head cushion can be ordered as extra equipment for extending the back cushion.*

## Unpacking/Installation

The upper square-section tube on the back cushion must be removed and replaced by the tube supplied with the head cushion (please see drawing for description).

## Technical specifications

Length:	38 cm
Width:	25 cm
Weight	1 kg

## FOLLO DIEM ARM ROTATION TRAINER

When ordering, please quote No. 136-028936

*This unit is specially designed for arm and shoulder rotation training. Installation is the same as for the foot cushion (No. 136-028930). If you already have this item, you will only require the upper section (No. 136-028937).*

### **Unpacking/installation:**

Take care when unpacking. **DO NOT** use a knife or other sharp instrument to avoid damaging the contents.

Make sure that the arm rotating trainer has not suffered transport damage.

Install the arm rotating trainer as shown on the drawing on the first page: Remove the plastic plug (as shown), install the arm rotating trainer's lock pin (5) in the hole where the plastic plug was. Tighten the lock handle (6).

### **Safety instructions:**

Study the different adjustments of the arm rotating trainer.

When making adjustments, make sure that there is no possibility of pinching. The specified max. weight (30 kg) must not be exceeded.

### **How to use:**

#### **Height adjustment:**

Loosen the screw (1). Depress the lock handle (2) and adjust to the required height. Make sure that the locking device is properly locked in position. Tighten the screw (1).

#### **Angle adjustment:**

Pull out the lock pin (3) on the top of the height adjustment tube and adjust to the correct angle. Make sure that the locking device is properly locked in position.

#### **Adjustment of hand support:**

Loosen the adjustment screw (4) and adjust to correct distance. Tighten the adjustment screw.

### **Maintenance instructions:**

#### **Check the following at regular intervals (every 3 months):**

Check that the arm rotating trainer is securely fastened to the bench and that the hinging and locking devices are functioning properly.

Inspect for damage for abnormal wear.

### **Technical specifications:**

Height: 52 → 85 cm  
Width: 20 cm  
Length: 42 → 60 cm  
Weight: 6.5 kg  
Maximum load: 30 kg

## FOLLO DIEM FOOT CUSHION

When ordering, please quote No. 136-028930

*This unit is specially designed to provide support for the feet. Attachment is the same as for the shoulder rotation trainer (No. 136-028936). If you already have this item you will only require the upper section (No. 136-028931).*

### **Unpacking/Installation**

Take care when opening the packaging. **DO NOT** use a knife or similar to cut open the packaging. This could damage the foot cushion. Inspect the foot cushion for transport damage.

Install the foot cushion as shown on the drawing. Remove the plastic plug as shown, fit the foot cushion inserting the locking stud (3) in the (plastic plug) aperture. Lock the foot cushion in position by tightening the locking handle (4).

### **Safety information**

Study the scope of adjustment for the foot cushion.

When adjusting, make sure that there is no risk of nipping anything/anyone.

Load on the foot cushion must not exceed the specified weight (30 kg).

### **Instructions for use**

#### **Height adjustment:**

Slacken the tightening wheel (1). Grip and depress the locking handle (2) and adjust to required height. Release locking handle (2) in the required position, making sure it is in the locked position. Tighten the wheel (1).

#### **Reversing the foot cushion:**

If you wish to reverse the foot cushion with the steel attachment the other way, loosen the wheel (1). Depress the locking handle (2) and withdraw the cushion from its holder. Reverse the cushion 180 degrees and replace in its holder. Adjust to required height, release locking handle (2), making sure it is in the locked position. Tighten the wheel (1).

### **Maintenance**

#### **Check the following at regular intervals (every 3rd month):**

Make sure that the foot cushion is securely attached to the bench, and that hinges and locking function are working properly. Inspect for damage or abnormal wear. Lubricate all hinges and threads with a drop of oil or smear with a little Vaseline.

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FOLLO FUTURA A/S