**EC DECLARATION OF CONFORMITY FOR
MACHINERY AND MEDICAL DEVICES**

Manufacturer: *Follo Futura AS*

Address: *Brekkeveien 43
1430 Ås
NORWAY*

Herewith declares that: *FOLLO DIEM PULLEY 20/40-EXPLOSIVE 50*

- is in conformity with the provisions of the Machinery Directive (Directive 89/392/EEC, 91/368/EEC, 93/44/EEC annex I) and Medical Devices Directive (93/42/EEC annex VII)

Class I.

Ås, Norway

Mette Veiby
Managing Director

Follo Diem weight based

SERIAL NO: _____

- Follo Diem pulley 20 kg 40 kg
- Follo Diem explosive
- Double Single
- Follo Diem mobile 20 kg 40 kg
- Follo Diem pull down 50 kg 80 kg

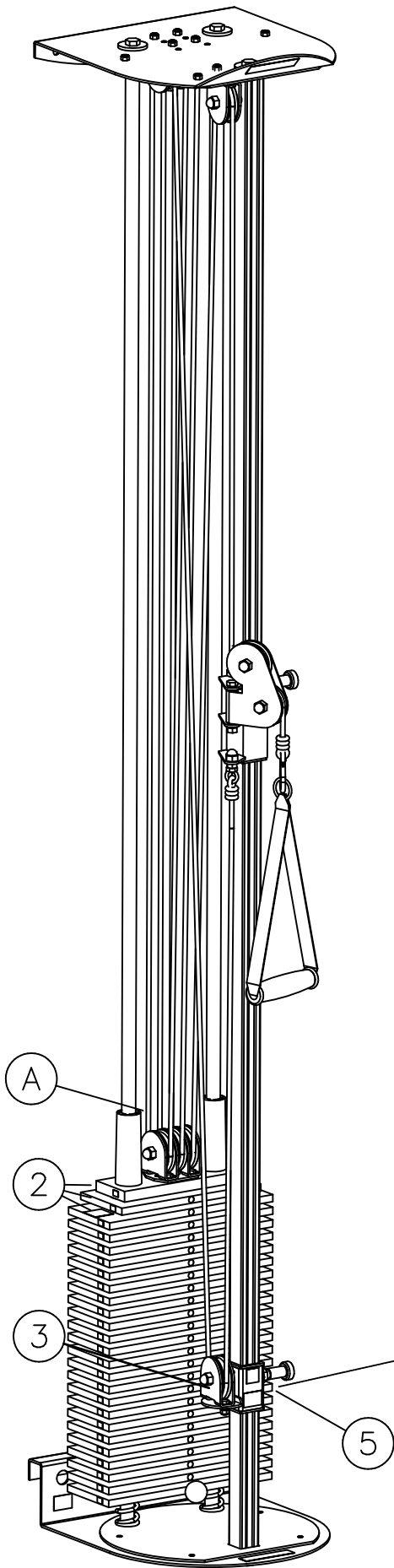
Enclosed:	Check point:
<input type="checkbox"/> Handles	<input type="checkbox"/> Weights
<input type="checkbox"/> Screws etc	<input type="checkbox"/> Rope system
<input type="checkbox"/> Instruction	<input type="checkbox"/> Adjustment
<input type="checkbox"/> Fireman hocks	<input type="checkbox"/> Surfaces
<input type="checkbox"/> Other	<input type="checkbox"/> Labels

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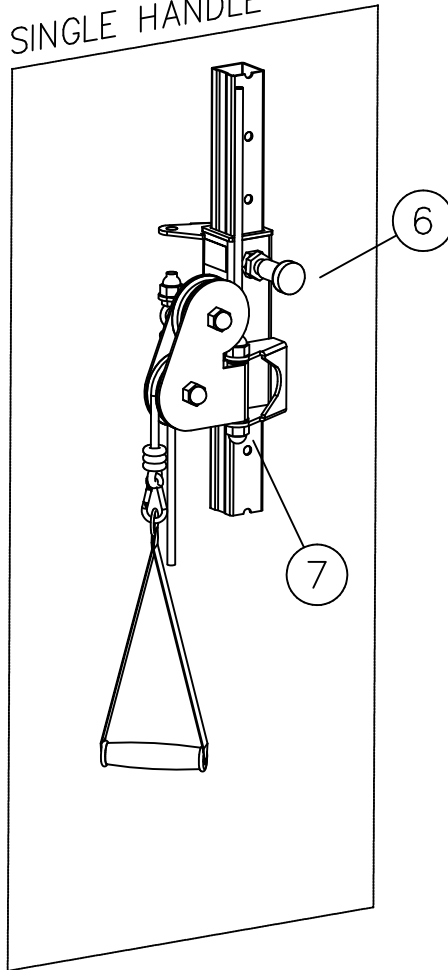
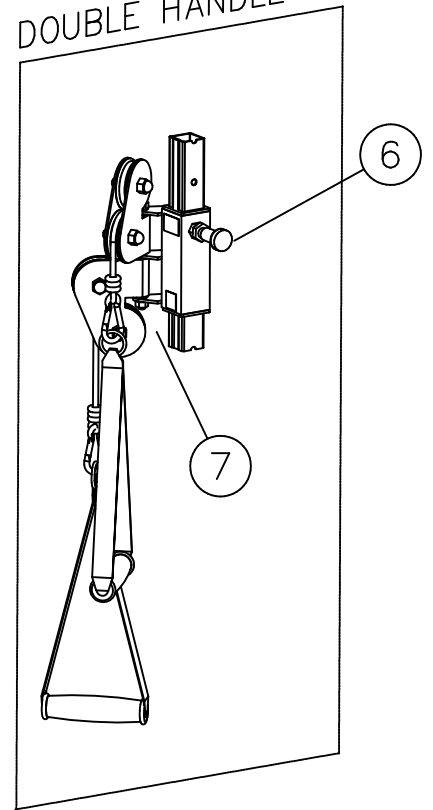
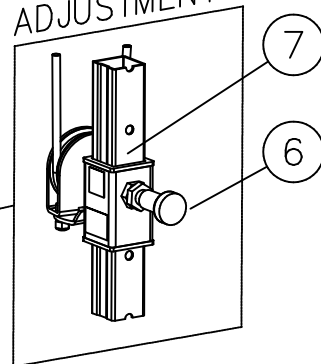
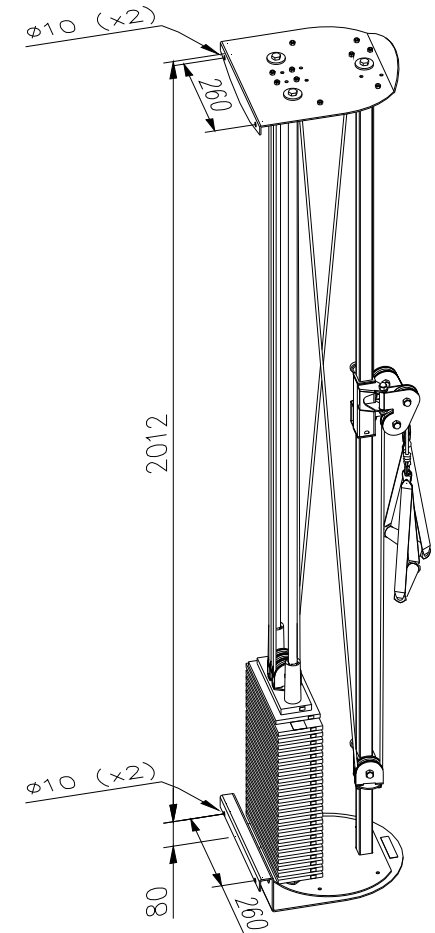
GB FOLLO DIEM EXPLOSIVE 50 KG

We congratulate you on choosing a Follo Diem product. This manual enables you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use. Contents:

- Unpacking/Installation
- How to use
- Technical specifications
- Safety information
- Maintenance instructions
- Spare parts list

UNPACKING/INSTALLATION:

1. Take care when removing packing materials. **DO NOT** use a knife or other sharp instrument to avoid damaging the contents.
2. Inspect the apparatus for transport damage.
3. Do not lift the apparatus by the guide tubes for the weights (1).
NOTE: The weights are locked in position with transport locking (A). This must be removed before using the apparatus.
4. The training pulley may be fitted on a wall, a modul system, or a freestanding frame.
NOTE: When fitting to a wall, make sure that the wall attachment points have sufficient hold. MINIMUM REQUIREMENT 1000N vertical tension from wall.

When installing on a mounting block or frame, follow the assembly instructions enclosed with the apparatus. Make sure that the apparatus is in plumb when installing, otherwise the functioning may be impaired.

SAFETY INSTRUCTIONS:

1. When adjusting height (4) and the cord elongation (5), make sure the blocking devices have clicked into position properly after adjustment.
2. When adjusting weight, make sure that the weight fork (3) is fully inserted.
NOTE: before adjusting the weight, make sure that the training pulley is not under load.
3. If the cord is visibly damaged it must be replaced before the apparatus can be used again.
4. Use the snap hook provided to attach other handgrips/slings etc.
5. The specified maximum weight must never be exceeded.

HOW TO USE:

1. Height adjustment of handgrip (4): → Pull out knob (6) and adjust to the required height (7). Make sure that the locking device is properly locked in position.
2. Cord elongation (5): → Follow the same adjustment procedure as above.
3. Weight adjustment: → To adjust the weight, insert the weight fork (3) in the required position. NOTE: To find the actual weight at the handgrip, divide the weight by 6 when using a single Explosive, and halve this again for Explosive double on one handle..

MAINTENANCE INSTRUCTIONS:

Check the following at regular intervals (every third month):

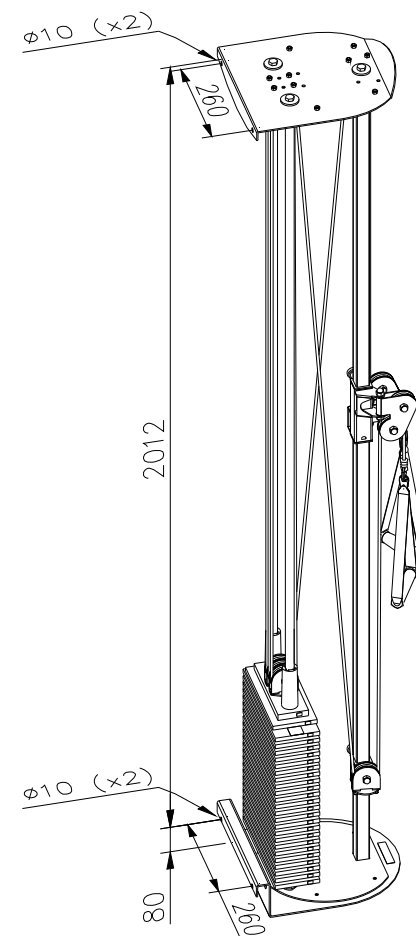
1. Wear and tear in cored and knotted ends.
The easiest way to replace the cord is to splice the ends of the old and new cords and use the old cord to pull the new cord through the pulleys.
2. Check attachment points in wall, mounting block or frame.
3. Make sure all the pulleys rotate easily.
4. Make sure that the locking devices for height adjustment and cord elongation are functioning properly.
5. Wipe down the guide tubes (1) and give them a thin coating of ordinary silicone spray.

TECHNICAL SPECIFICATIONS:

Height: 216cm
Width: 33cm
Depth: 37cm
Weight: 70kg

ACCESSORIES:

- Triceps bar with or without rotating attachment point
- Rowing handles
- Leather slings
- Latitimus handgrip
- Hand support
- Pronation/supination handgrip

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