
FOLLO DIEM PULL DOWN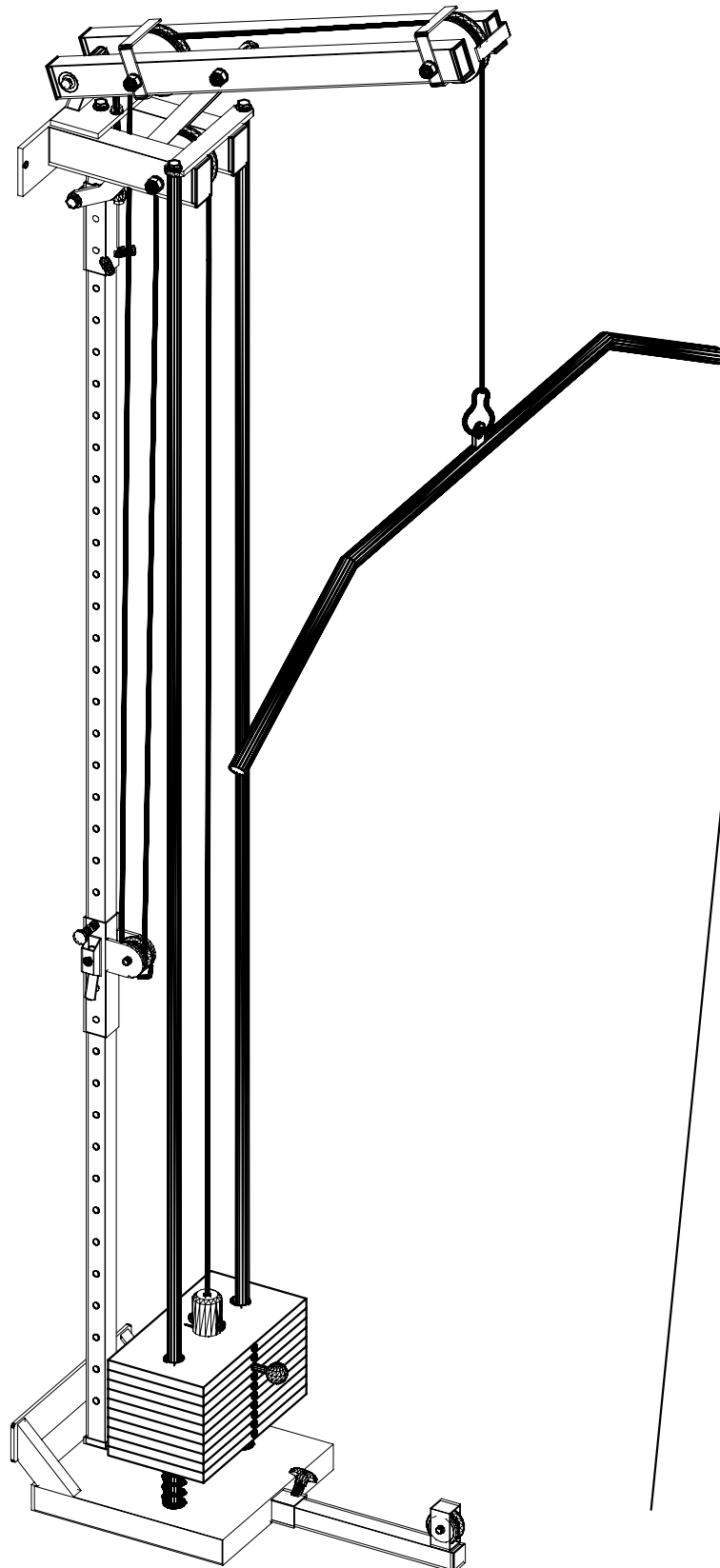

EC DECLARATION OF CONFORMITY FOR
MACHINERY AND MEDICAL DEVICES

Manufacturer: *Follo Industrier AS*

Address: *Brekkeveien 43*
1430 Ås
NORWAY

Herewith declares that: *Follo Diem Pull Down 50kg/80kg (131-012912...)*

- is in conformity with the provisions of the Machinery Directive (Directive 89/392/EEC, 91/368/EEC, 93/44/EEC annex I) and Medical Devices Directive (93/42/EEC annex VII)

Class I.

Ås, Norway

Ingar Bjerklund
Managing Director

Follo Diem weight based

SERIAL NO: _____

☐ Follo Diem pulley ☐ 20 kg ☐ 40 kg

☐ Follo Diem explosive

☐ Double ☐ Single

☐ Follo Diem mobile ☐ 20 kg ☐ 40 kg

☐ Follo Diem pull down ☐ 50 kg ☐ 80 kg

Enclosed:	Check point:
☐ Handles	☐ Weights
☐ Screws etc	☐ Rope system
☐ Instruction	☐ Adjustment
☐ Fireman hocks	☐ Surfaces
☐ Other	☐ Labels

ASSEMBLED (date/sign.): _____

END CONTROLLED
(date/sign.): _____

PACKING CONTROL:
(date/sign.): _____

FOLLO FUTURA AS - POSTBOKS 112 - 1430 Ås - NORWAY


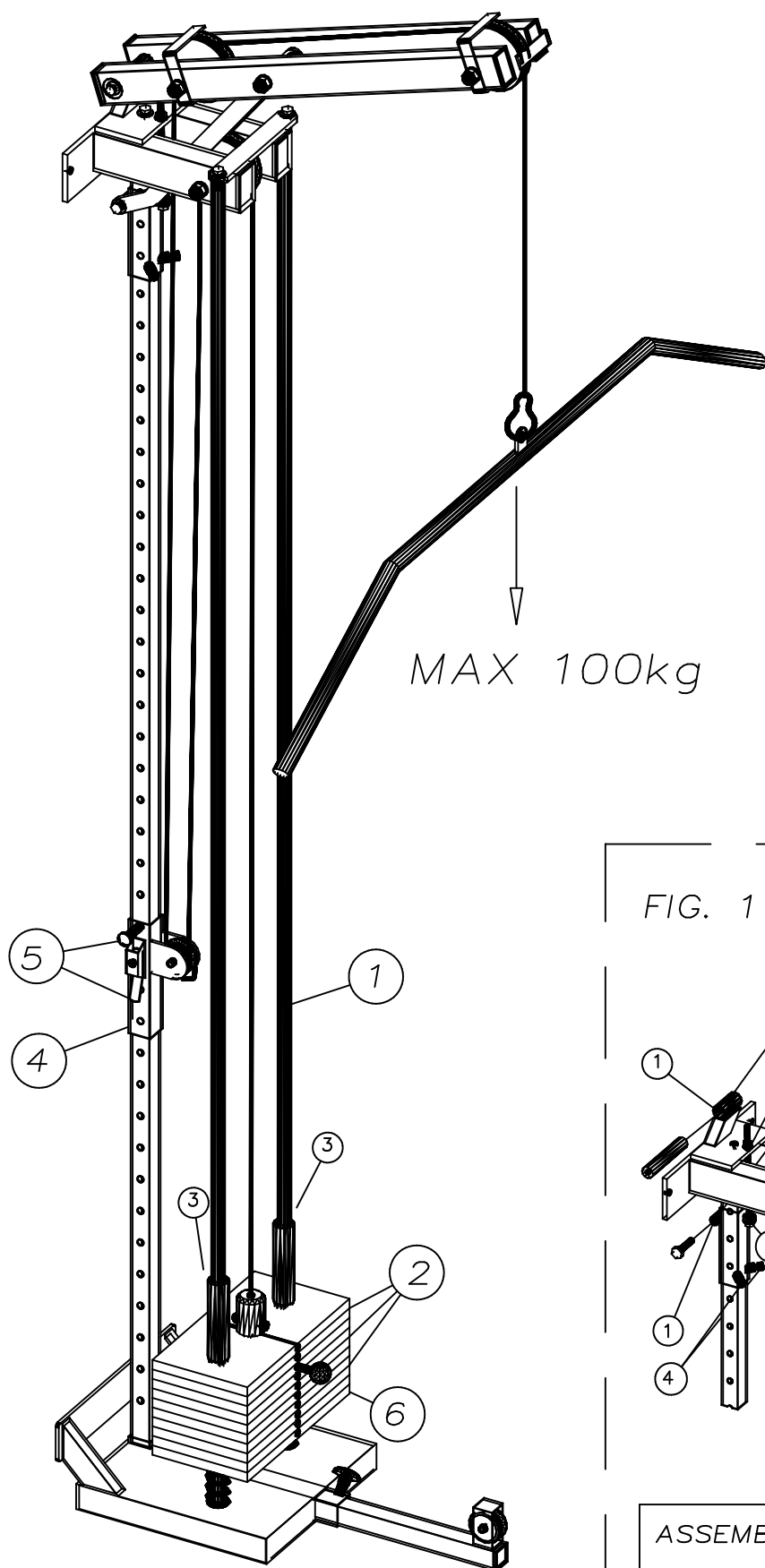
FIG. 1

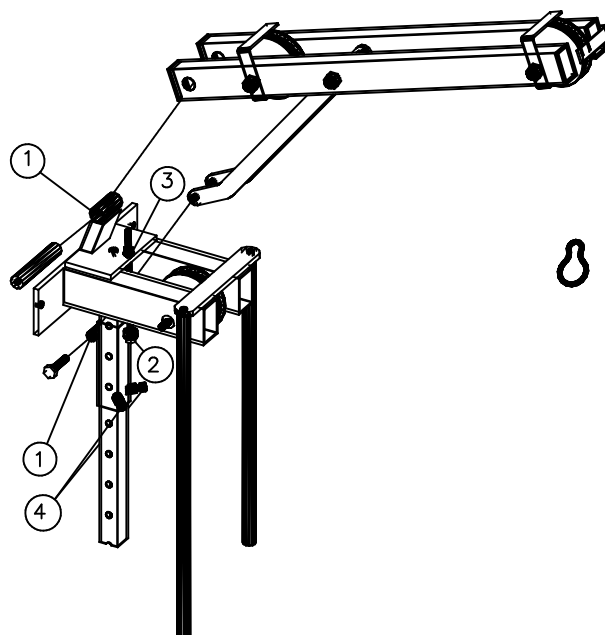ASSEMBLY FOR TOP FRAME

- ① ENTER UPPER AND LOWER BOLTS/NUTS BEFORE TIGHTEN
- ② HIGH/LOW ADJUSTING SCREW TOP FRAME
- ③ TIGHTEN THE TRANSPORT LOCKING
- ④ TIGHTEN SECURE SCREWS

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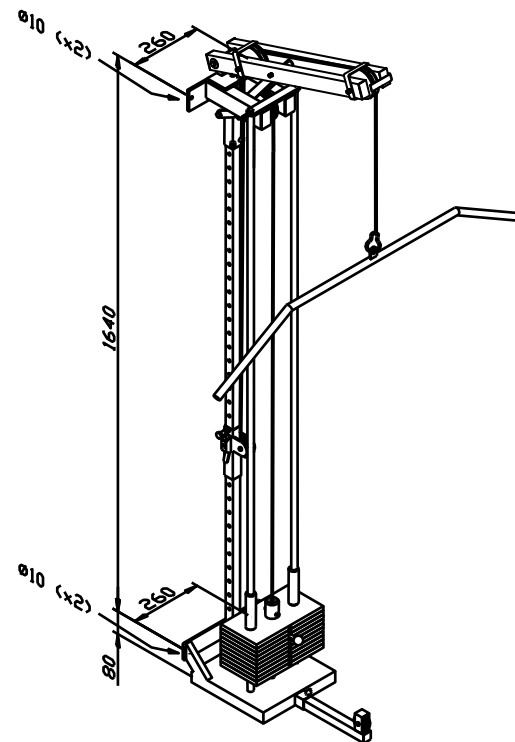
GB FOLLO DIEM PULL DOWN

We congratulate you on choosing a Follo Diem product. This manual enables you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use. Contents:

- Unpacking/installation
- Safety information
- How to use
- Maintenance instructions
- Technical specifications
- Spare parts list

UNPACKING/INSTALLATION:

1. Take care when removing packing materials.
DO NOT use a knife or other sharp instrument to avoid damaging the contents.
2. Inspect the apparatus for transport damage.
3. **Do not** lift the apparatus by the tubes for weight (1).
4. Make sure that the weight (2) are resting on the bottom plate before raising to the upright position.
NOTE: The weights are locked in position with transport locking (3). These must be removed before using the apparatus.
5. The apparatus may be fitted on a wall or a mounting block.
NOTE: When fitting to a wall, make sure that the wall attachment points have sufficient hold. MINIMUM REQUIREMENT 1000N vertical tension from wall.
6. For height adjustment on top frame, please read the enclosed instruction "Assembly for top frame" (Fig. 1) No. 2 to 4.

When installing on a mounting block or a frame, follow the assembly instructions enclosed with the apparatus. Make sure that the apparatus is in plumb when installing, otherwise the functioning may be impaired.

SAFETY INSTRUCTIONS:

1. When adjusting the cord elongation (4), make sure the blocking devices (5) are properly clicked into position after adjustment.
2. When adjusting weight, make sure that the locking pin (6) is fully inserted.
NOTE: Before adjusting the weight, make sure that the pull-down is not under load.
3. If the cord is visibly damaged it must be replaced before the apparatus can be used again.
4. The specified maximum weight must never be exceeded.

MAINTENANCE/INSTRUCTIONS.

Check the following at regular intervals (every third month):

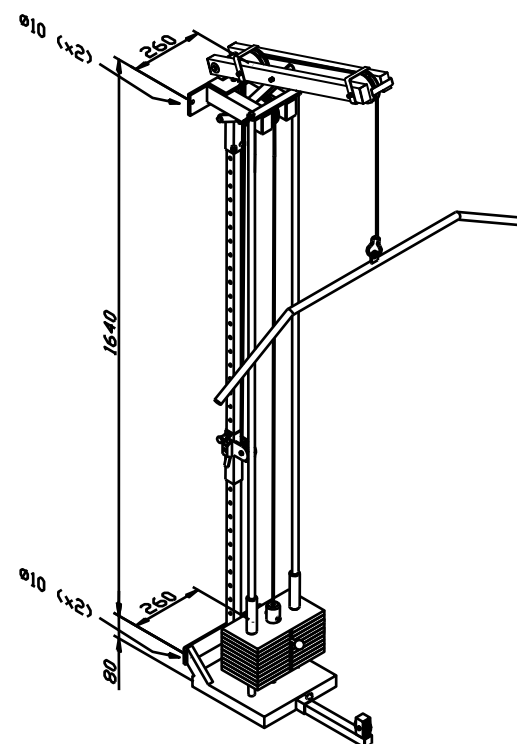
1. Wear and tear in cord and knotted ends.
2. Check attachment points in wall, mounting block or frame.
3. Make sure all the pulleys rotate easily.
4. Make sure that the locking device for cord elongation is functioning properly.

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FOLLO FUTURA A/S