FOLLO STABILIS MANUPED

FOLLO  FUTURA

We congratulate you on choosing a Follo Stabilis product. This instruction information makes it possible for you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.
Contents: Check list, unpacking/Installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list

ORDER NO.:	
COLOUR:	
SERIAL NO.:	

FUNCTION TEST:	<input type="checkbox"/> Surface: No damages
<input type="checkbox"/> Bottom frame: Wheels, Screws, plastic parts	<input type="checkbox"/> Labels: Serial number
<input type="checkbox"/> Crank: Securing the pin hole	
<input type="checkbox"/> Top frame: Surface, screws, plastic parts	ENCLOSED:
<input type="checkbox"/> Gas spring: Height adjustment, lock nut	<input type="checkbox"/> Instructions
<input type="checkbox"/> Adjusting handle	<input type="checkbox"/> Screws, straps

ASSEMBLED (date/sign.):	
-------------------------	--

END CONTROLLED (date/sign.):	
------------------------------	--

PACKING CONTROL: (date/sign.):	
--------------------------------	--

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The purpose of the training

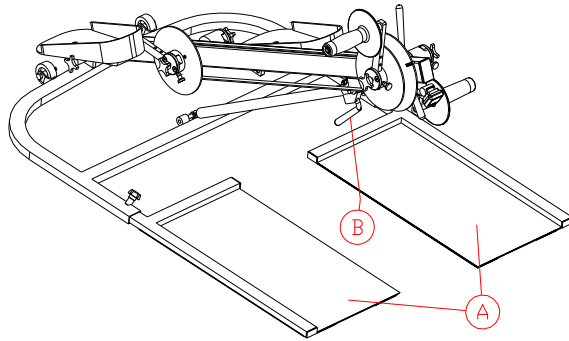
The Manuped machine is designed to improve circulation. It can be used standing on the floor (as shown in the illustration on the cover) while the patient sits on a chair/wheelchair, which has been placed on the floor plates (to keep the machine steady and in position during use). Select a resistance that the patient can keep up for at least 15 minutes. Start pedalling using even movements, extending movements as far as possible. Read off time and distance on the counter.

Safety information

- Make sure that nothing can be pinched or jammed and that the handles and pedals rotate easily and are properly secured. Avoid using very loose clothing, which could get caught up in the belt, handles or pedals.
- Always test the machine's functions yourself, and allow the patient to test all functions under supervision before handing out the product for self-training.


Unpacking

3. Take care when removing the machine from its packaging. Do **NOT** use a knife or other sharp instrument as this may damage the machine. Inspect the machine for any transport damage.
4. Place the machine on the floor as shown in the drawing. Stand on the floor plates A) and press in the lever B) to release the gas-assisted spring. The gas-assisted spring will now push the machine upwards at low pressure. See also item 5 GUIDELINES FOR USE.


Guidelines for use

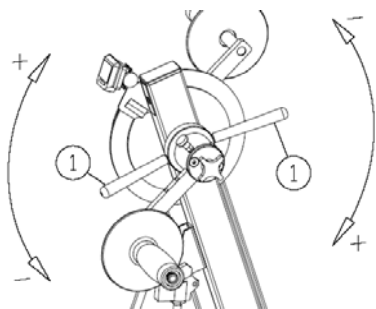1. Use: The Manuped is designed for use standing on the floor while the user sits on a chair/wheelchair placed on the floor plates. Make sure that the chair is pushed well onto the floor plates.

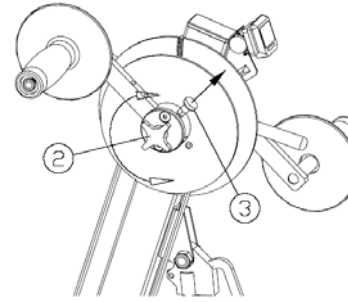
2. Adjusting resistance: Adjust resistance using the levers (1). Rotating the levers in a clockwise direction increases resistance, and loosening anti-clockwise reduces resistance.


3. Adjusting the length of handles and pedals: The crank throw of the handles and pedals may be adjusted by first loosening the knob (2). Pull out the lockpin (3) and push or pull the handle or pedal to the required position. Make sure that the lockpin (3) lines up with a hole and replace the lockpin. Tighten the knob (2).

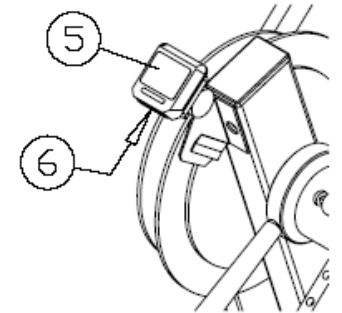
4. Counter: The counter (5) is a bicycle computer. It registers when training starts, and is switched on automatically (at the same function as when it was last used). Use the button (6) on the bicycle computer to switch between the different functions: Distance, time, speed, clock and auto (changes regularly between all functions). Holding down button (6) will clear time and distance (not the clock). The computer will switch off automatically when not in use for a period. Read more about the bicycle computer's functions in the separate instructions for use.

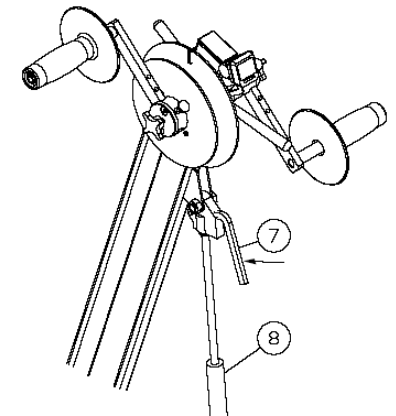
5. Adjustment of upper section: The upper section of the Manuped can be adjusted to the required position (or folded away) by pressing down (7) at the top of the gas-assisted spring (8) and at the same time holding the top of the machine near the counter (the gas-assisted spring will push the section upwards at low pressure). By pushing downwards or releasing, the position of the unit can be adjusted up or down. When you have found the best position, release handle (7) and the machine is ready for use.

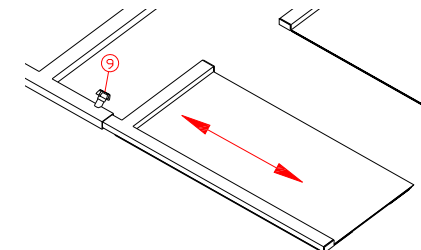6. Floor plates: Adjustable inwards and outwards by loosening knob (9). Adjust to required position and tighten knob (9).

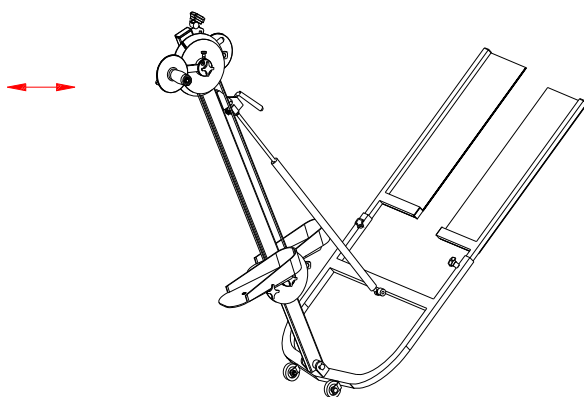7. Moving / transporting the machine: When the machine is to be moved or re-positioned indoors, the easiest method is to move it on its transport wheels. Tilt the machine forwards onto its wheels as shown in the drawing. The machine can then be moved and positioned with ease. If the machine is to be transported over longer distances, it is easy to fold up. Push the floor plates fully in and fold down the main beam as shown in the illustration in the UNPACKING section. The machine takes up little space and is easy to carry.

Maintenance

Inspect the machine regularly for damage, play or missing parts. In the case of damage or other deficiencies, the machine must not be used until it has been repaired.

Troubleshooting

- Counter / computer:
If the counter stops working (empty display), check the battery first. If the counter is working, but does not count, check that the black transmitter on the largest nickel plated disc (No. 44 on the parts drawing) is in position. Check also that the opening between the black transmitter and the sensor (immediately below the computer) is correct when they pass each other. The opening must be between 2 and 4 millimetres (maximum 5 millimetres). Adjust the opening by carefully bending the baseplate to which the sensor is attached. See also the folder that accompanied the computer.
- Brake resistance:
If no brake resistance is felt or the resistance is not functioning properly, the handle may have been adjusted too far in the wrong direction (several turns). Turn the handle clockwise until heavy resistance is felt. Then adjust to the required resistance. If resistance is uneven, inspect the brake lining (No. 46 on the parts drawing). If the brake lining has loosened, it must be replaced.
- Gas-assisted spring:
If the gas-assisted spring in the adjustment strut is not functioning properly, try giving it manual assistance (up or down). This may help if the spring has seized in position.

If the gas-assisted spring does not lock in position when the lever is released, the spring must be adjusted. Unscrew the gas spring's upper attachment point (Nos. 58+29 on the parts drawing). Loosen the nut on the gas spring strut (immediately below the lock/unlock handle) and move the attachment (No. 28) upwards slightly until the lever is functioning properly.

If the gas-assisted spring does not function when the lever is depressed, proceed as described above, but move the attachment point (No. 28) in the opposite direction to adjust.

If these guidelines fail to rectify the situation or if there are other faults or deficiencies, contact your dealer for replacement parts or for service.

NB! The manufacturer responsibility for CE marking is invalidated if any form of modification is carried out.

Extra equipment

- Comfort discs for handles
- Gloves for handles
- Large pedals

Technical specifications

Weight with/without floor plates:	20,9 / 14,5 kg	Width:	77 cm	Length with/without floor plates:	114 / 56 cm
Height:	110 cm				

Recycling

This machine is comprised mainly of steel, rubber and plastic. When separating for recycling, the handles, pedals, drive belt and plastic covers must be handled as hazardous waste. The remaining parts of the machine can be handled as metal.

The "CE" mark indicates that this product conforms with European standards for health, safety and the environment.

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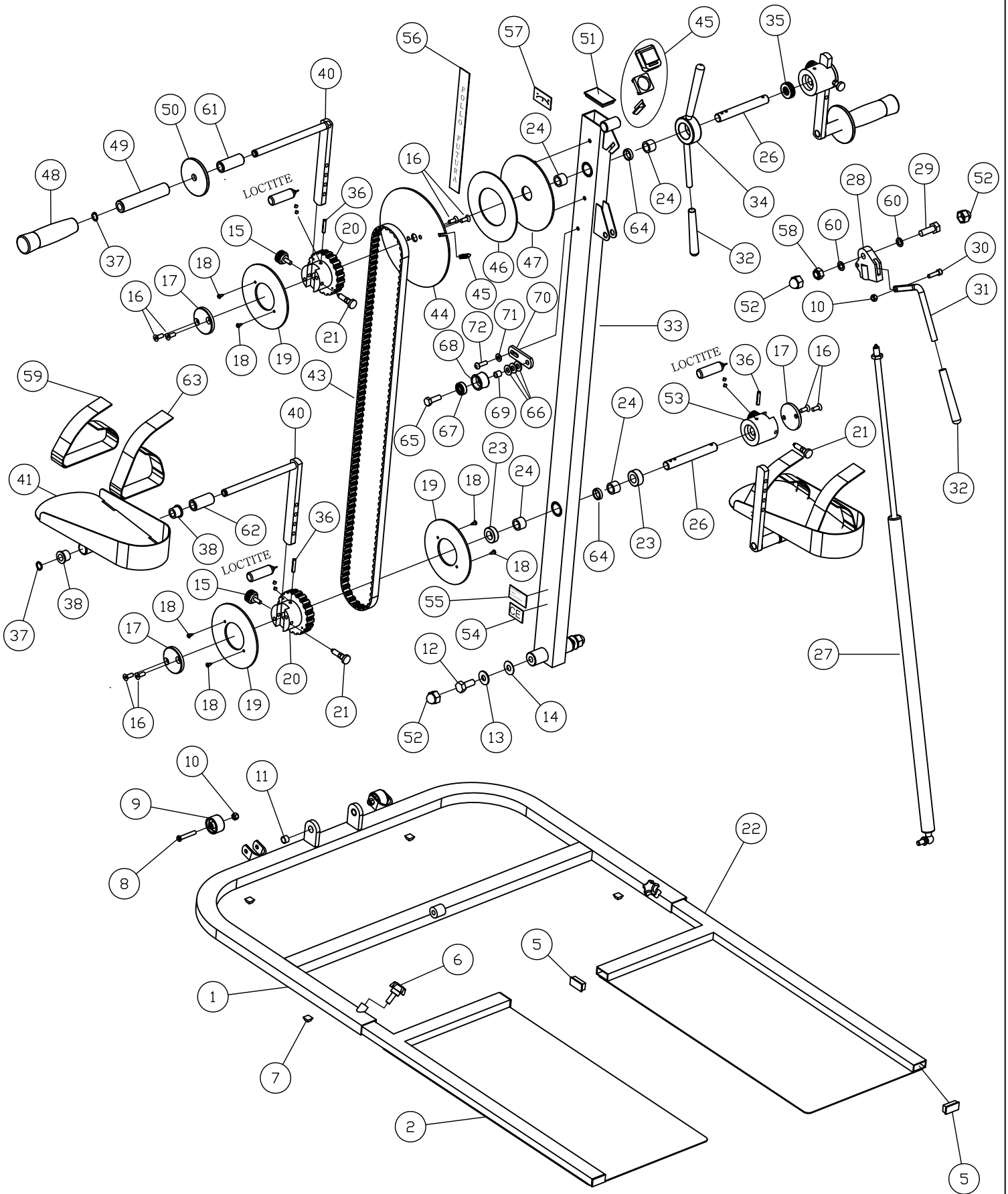
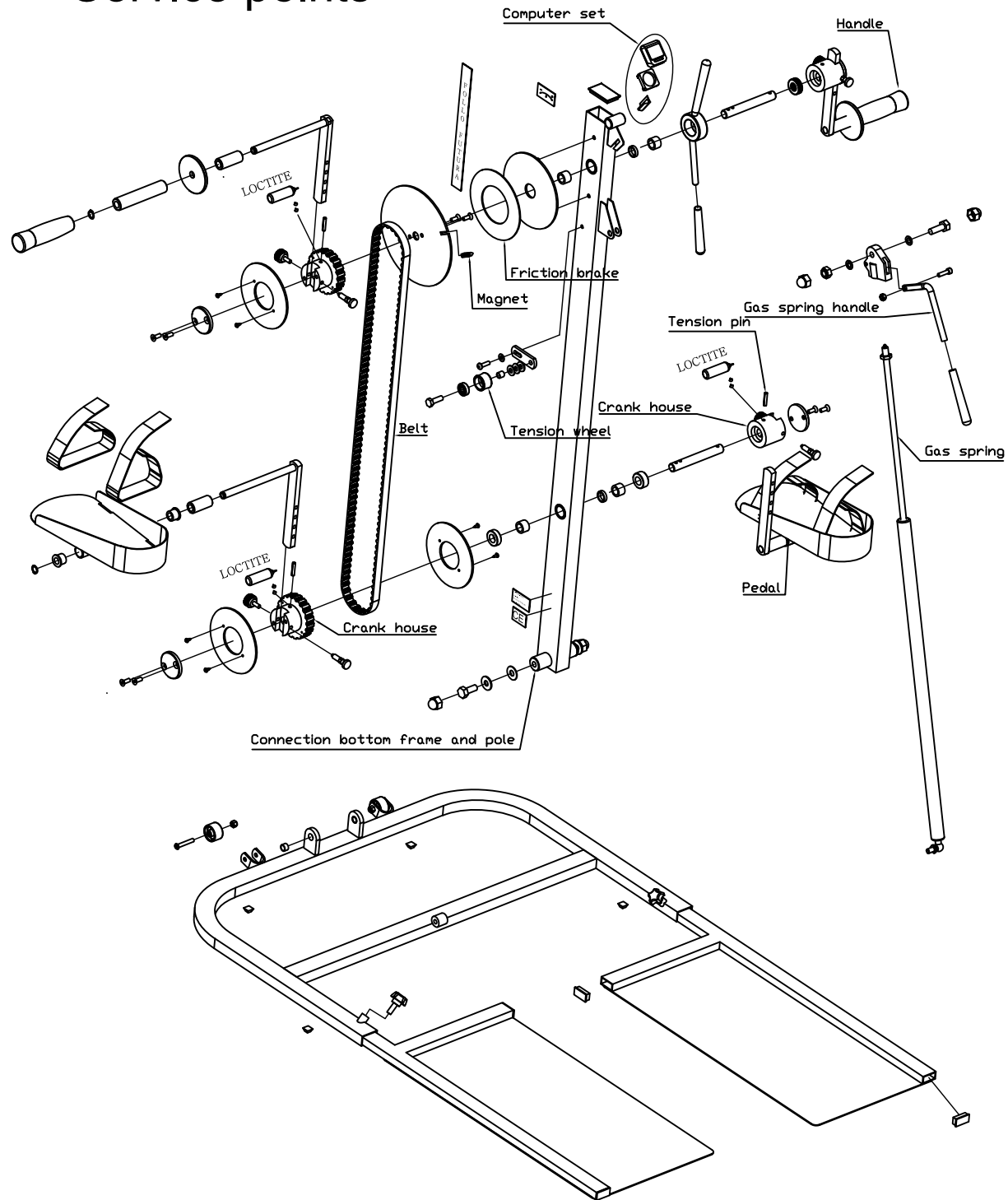
Service points

1. Check the friction brake and if it is worn, change it.
2. Make sure the handles and pedals are rotating easy.
3. Check the tension pins on the crank housings.
4. Make sure the gas spring and gas spring handle are working properly.
5. Check the computer parts and the magnet.
6. Check the connection between bottom frame and the pole.
7. Check the belt for wear and cracks.
8. Check the tension wheel.
9. Check out all screws, nuts and parts.
10. Do not use alcohol based agents on the friction brake.
11. Need the paint to be improved, use color RAL 9006