



CLICK

2018 SOLUTIONS GUIDE

Welcome to the Solutions Guide

from Click Medical



Click Medical develops solutions that make it simple for practitioners, technicians or central fabricators to integrate advanced adjustment technology into custom orthotic and prosthetic devices.

Users experience better fit, performance and day-long comfort from their medical devices. Practitioners provide better devices with a broad range of fit, which reduces office visits.

**Join us in changing
millions of lives with
2 fingers and a twist!**



CLICK

Click Medical
1205 Hilltop Parkway, W101
Steamboat Springs, Co 80487



Want to chat? 970.670.7012 or info@clickmedical.co



NEW QuickFit™ Lacer

With QuickFit™ Lacers, you can add on easy one-handed adjustability to dial in fit.



| SKU | Item No. | Size | Details |
|---|---------------|--------|---|
|  QuickFit™ Lacer* | OK0200-130-05 | Small | Designed for footwear size 41 EU and smaller. Comes with a low-profile dial and 6 guides. Rivets sold separately. |
| | OK0220-130-05 | Medium | Designed for footwear size 42 EU and larger. Comes with a low profile dial and 8 guides. Rivets sold separately. |
| | OK0240-220-05 | Large | Designed for AFO's and above the ankle boots. Comes with a more powerful dial and 10 guides. Rivets sold separately. |
|  QuickFit™ Setter Tool | TL001 | | Recommended installation tool. Customized for use with QuickFit Rivets. Properly secures rivets and provides appropriate hole-punch for creating rivet holes when needed. |
| QuickFit™ Rivets 100 pack | SP1040 | | Available in packs of 100. Rivet pack is required to install any QuickFit Lacer Kit. Rivets are not included with QuickFit Lacer Kits. |

*Lacers are sold individually. One pair of footwear requires two kits (one per shoe).

RevoFit™

Adjustable Socket Kits



EASY DIAL-IN FIT

Easily customize the fit, feel and performance of the prosthesis by dialing-in the socket fit.



CUSTOM PANEL DESIGN

Prosthetists can create custom panels targeted to adjust specific areas of the limb that fluctuate in volume while leaving sensitive areas undisturbed.



CONVENIENT

- Donn & doff easily
- Replaces sock management
- Provide a broader range of fit




FIT & COMFORT ON DEMAND

ADJUST SOCKET
WITH 2 FINGERS
& A TWIST!

RevoFit™

Our RevoFit™ line of prosthetic solutions enable prosthetists to fabricate micro-adjustable prosthetic sockets. Once integrated, the socket can be instantly customized, by simply turning the dial to create on-demand, targeted adjustability.

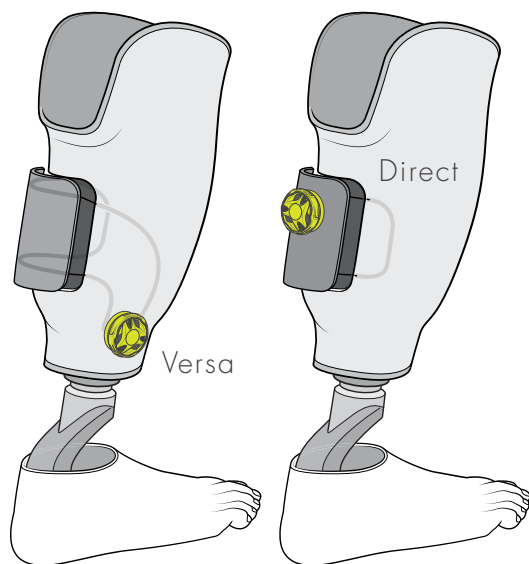

RevoFit™ Versa

RevoFit™ Versa allows for various placement options. The filament enters the base of the reel in parallel and can be placed anywhere on the socket.

#PK0150-320-05




RevoFit™ Direct


The RevoFit™ Direct filament enters the dial at 180 degrees which is ideal for quickly creating volume in the socket for easy donn/doff.

#PK0100-320-05


Create custom panels in variety of ways using Direct and/or Versa kits.

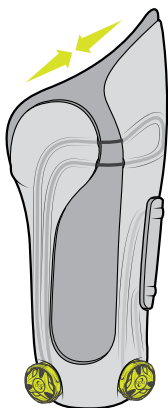

AK Two Panel Design
and Proximal Strap
with two Versa Kits

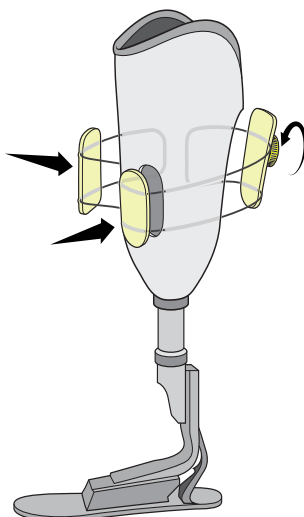

BK Three Panel Design
with Direct Kit

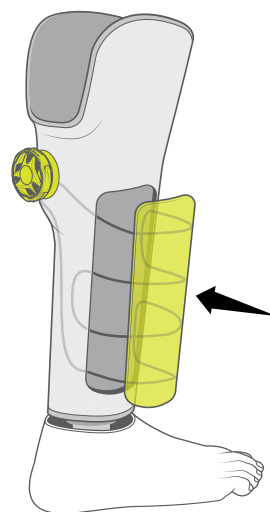

Symes Suspension
with Versa Kit

RevoLock™ Lanyard



EASILY DRAW LIMB INTO POSITION

RevoLock™ provides 2:1 mechanical leverage giving the user the ability to draw in the liner. This is ideal for patients with limited strength and/or dexterity.



LOCKED & HIDDEN CORD

RevoLock™ winds up the length of the lanyard cord while drawing the liner into the socket. The cord is always neatly stored and protected inside the dial.



STANDARD FABRICATION


RevoLock™ is fabricated into a socket using traditional lamination techniques and is straightforward to assemble and fabricate.

> ADDITIONAL BENEFITS

- Approved for L5671
- Extremely lightweight with low build height
- Sealed connection prevents airflow and can be used with suction systems
- Waterproof
- Adjustable through clothing
- Compatible with RevoFit™ adjustable sockets

RevoLock™ Lanyard

#PK3000-220-05

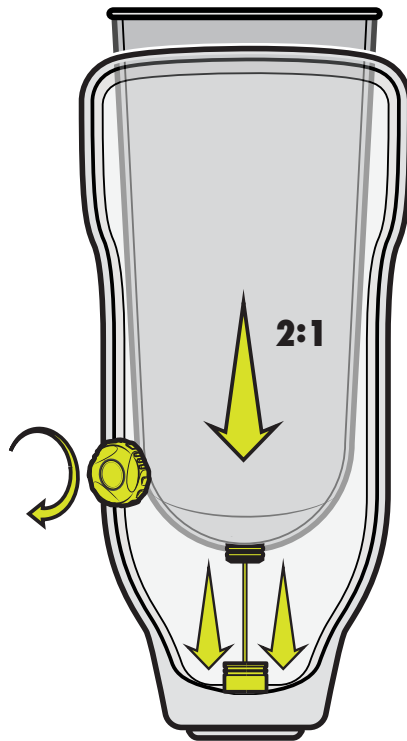


- > Easily draw liner into socket with 2:1 mechanical advantage
- > Low build height under 1"
- > Donn while seated
- > Mount dial anywhere

Available for lower and upper extremity




Shown:
RevoLock™ #PK3000-220-05

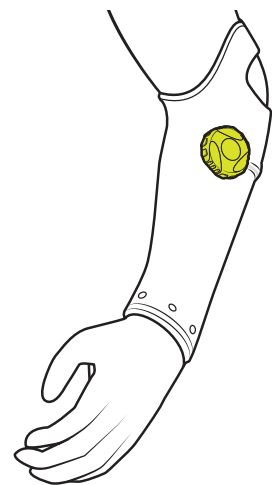

Shown:
• RevoLock™ #3000-220-05
• Upper Extremity Thread insert #PK1006-002

RevoFit™ & RevoLock™ Lanyard Kits

Adjustable Diagnostic Sockets

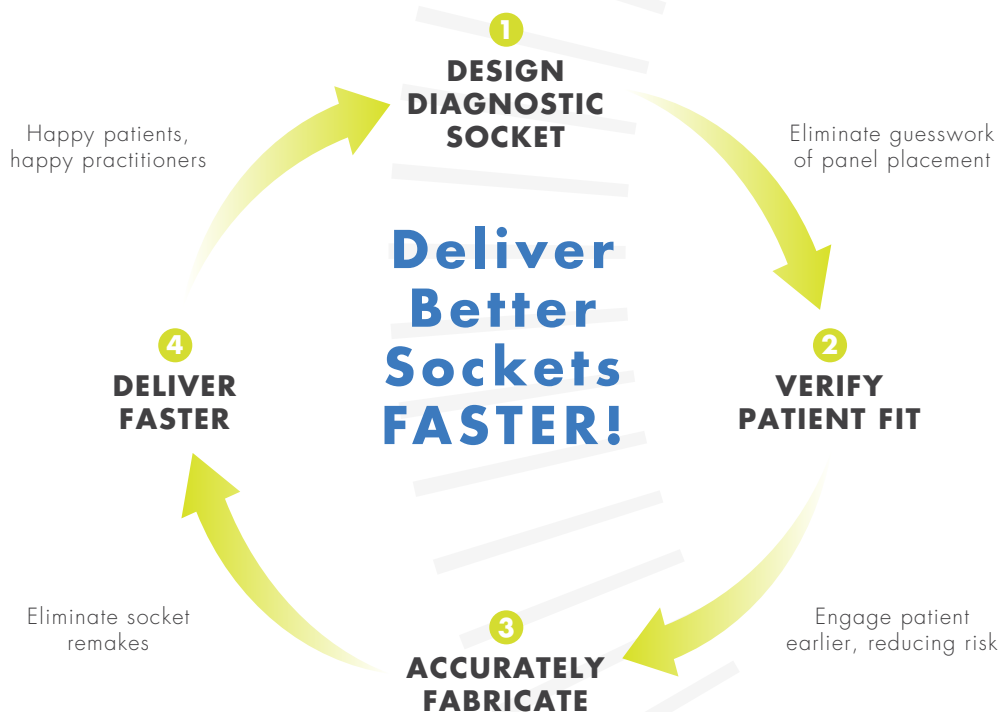

Diagnostic Compatible



All RevoFit™ and RevoLock™ Kits are designed for use in both diagnostic and definitive socket fabrication




Easily secure Revo to diagnostic socket with casting tape



Assess Revo function and design on your patient before fabrication of a definitive socket



Save Time & Money

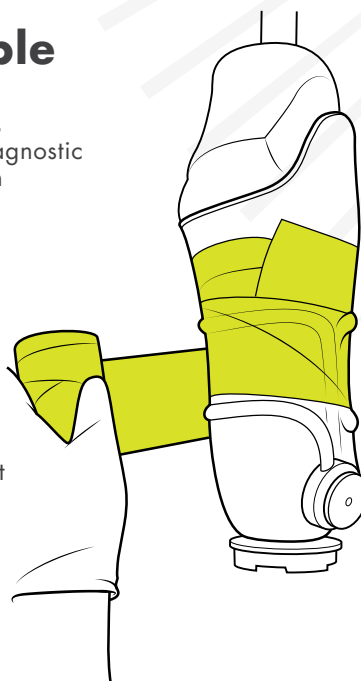

RevoFit™ Versa
#PK0150-320-05



RevoFit™ Direct
#PK0100-320-05




RevoLock™ Lanyard
#PK3000-220-05

RevoFlex™

Adjustable Dorsiflexion

We believe that adjustability is core to a patient being able to progress quickly through therapy and safely achieve mobility.

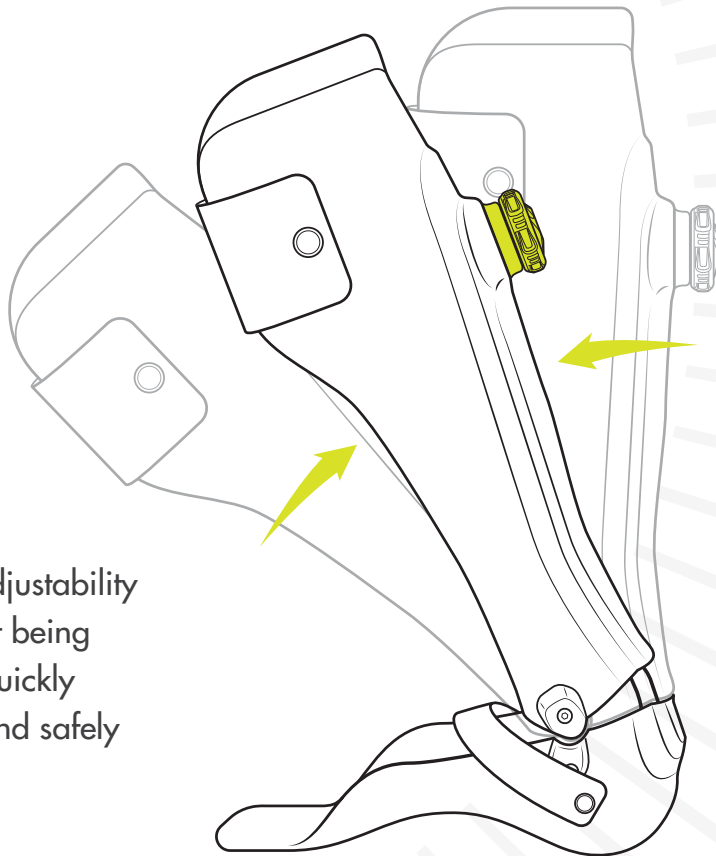

RevoFlex AFO Kit
#OK1100-320-05

Adjustability & Control



Adaptable Range of Motion

Easily adjust range of motion as patient progresses through therapy



On-Demand Adjustment

Control range of motion throughout the day as support needs change



Save Time & Money

Using one adjustable AFO reduces fabrication costs and delivers instant adaption to patient need



Instant adjustability of
plantar/dorsiflexion

OrthoFit™ Lacer Solutions



Click Medical Ambassador Max Conserva tests OrthoFit™ solutions in his gym in San Francisco

Creates great fit, comfort and performance on AFO's and Cuffs.



MICRO-ADJUSTABLE

Each click delivers 1 millimeter of precision fit and adjustment




CUSTOMIZABLE FIT

Delivers smooth, even closure with no pressure points for maximum comfort




EASY ONE HANDED OPERATION

The OrthoFit™ Lacer Solution is easy to use with one hand

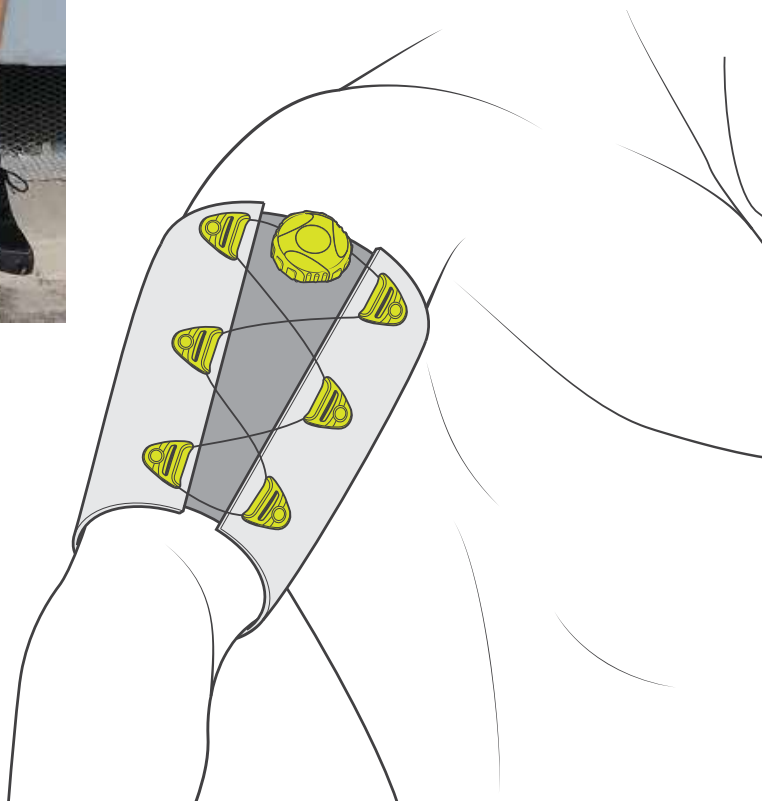

> **OrthoFit™ Lacer kit:**


Black Small: #OK0100-120-05
Black Medium: #OK0100-125-05
Black Large: #OK0100-220-05
Tan Large: #OK0100-220-20
Brown Large: #OK0100-220-15

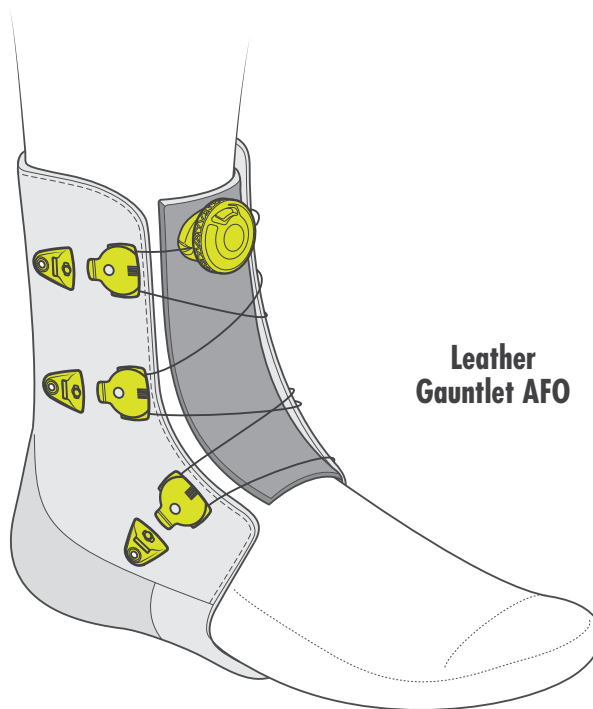

**Leather
Gauntlet AFO**

> **Optional Releasing Kit**

Add to any lacer kit to allow for
lacer to fully open during donn/doff.
#OK0150-100-05

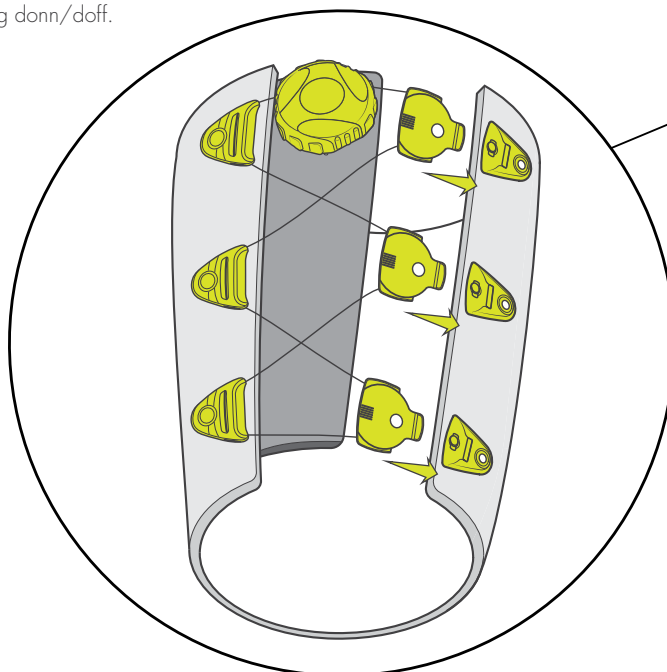

Guides release
to fully open.


QuickFit™ Straps

Say hello to a GIANT leap forward in closure & adjustment systems with the new QuickFit™ strap system.




Our QuickFit™ Straps require only one hand to close, lock & adjust.


Automatic Magnetic connector

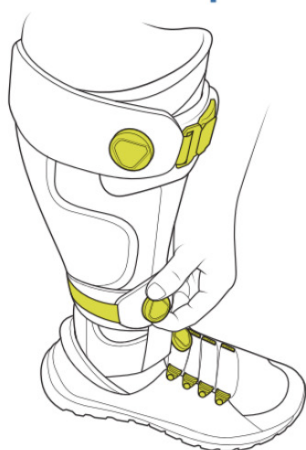The All New QuickFit Strap:



- One- Hand Adjustment
- Low Profile Wraps to Small Circumference
- Fit + Comfort On-Demand
- Quickly Add Adjustability

-  Push to Engage
-  Twist to Tighten
-  Pop Up for Quick Release

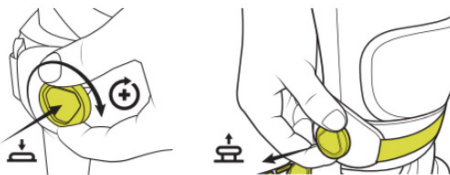

- OK1400-130-05 (QuickFit Strap 1", Black)
- OK1410-130-05 (QuickFit Strap 1.5", Black)
- OK1420-130-05 (QuickFit Strap 2", Black)

Add QuickFit Buckle to QuickFit Strap for Added Functionality


- Robust Buckle Does Not Wear Out
- Magnet Power Allows One Hand Operation
- Low Profile Buckle Fits Snag Free




1. Magnetic Self Connecting Buckle



2. Pull Tab to Release Buckle

- OK1405-000-05 (QuickFit Buckle 1", Black)
- OK1415-000-05 (QuickFit Buckle 1.5", Black)
- OK1425-000-05 (QuickFit Buckle 2", Black)

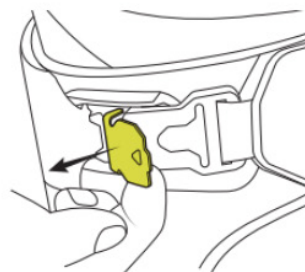

ClickFit™ Dynamic Straps


Dynamic Straps allow for even tension between the upper and lower straps. This dynamic adjustment provides great fit and reduces migration.

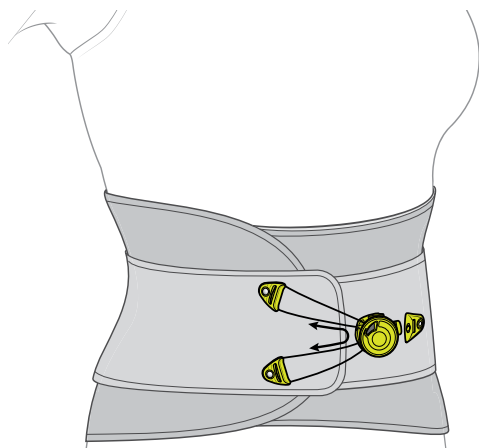


ClickFit™ Dynamic Strap Kit:

Small: #OK0210-120-05


Medium: #OK0210-125-05

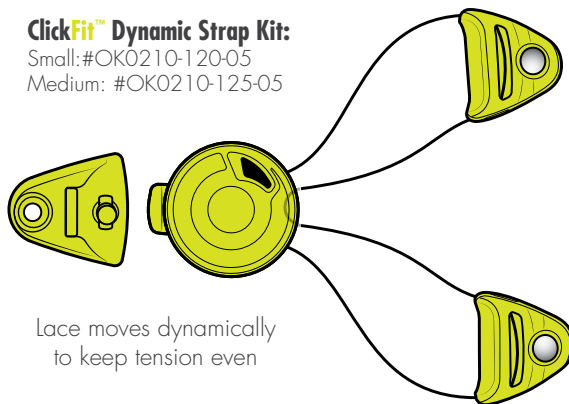

Lace moves dynamically
to keep tension even

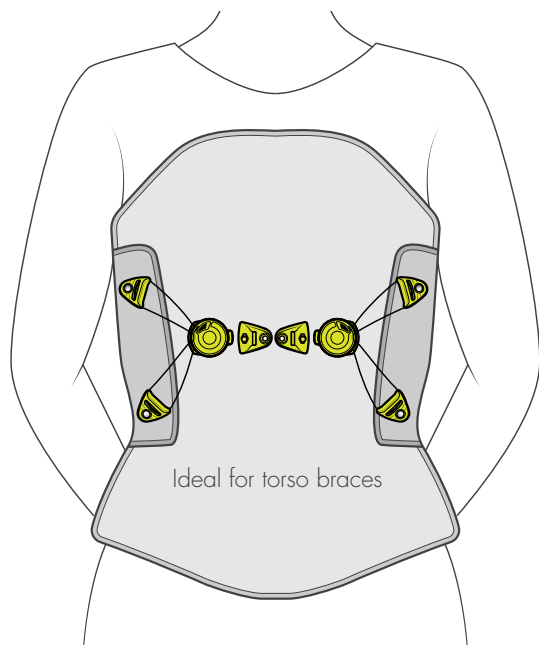

Ideal for torso braces

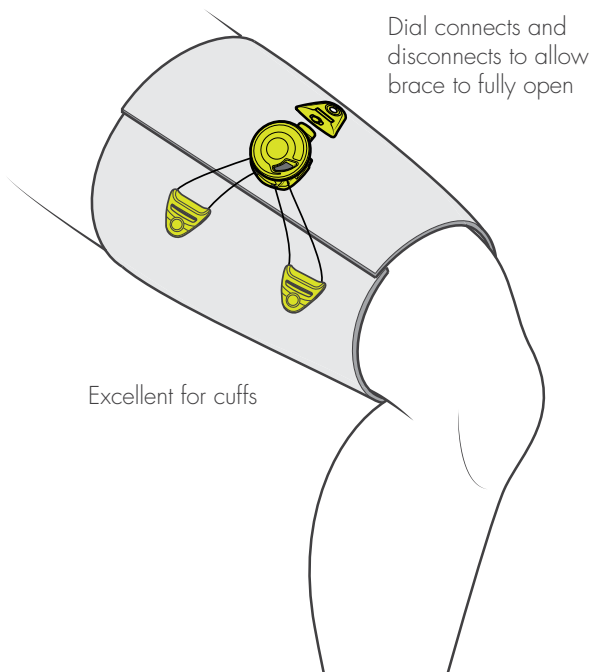


Dial connects and
disconnects to allow
brace to fully open

Excellent for cuffs

FootFit™


Tap into the true performance of
Shoos with the FootFit™

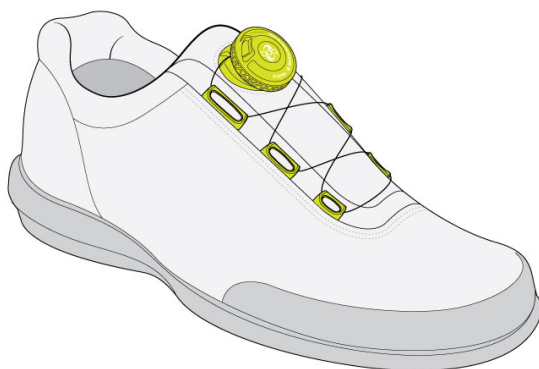

> FootFit™ kit:

Black Medium L4: #FK0120-125-05

Black Large M3: #FK0120-220-05

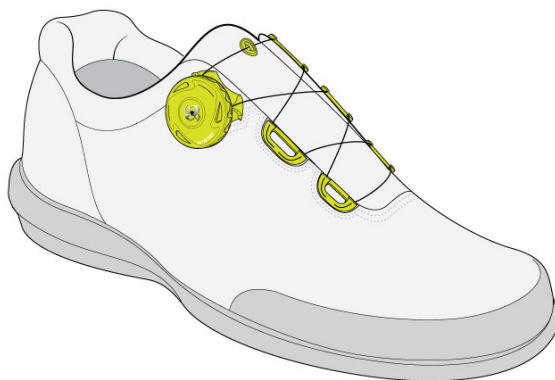

> FootFit™ kit:

Black Medium L4: #FK0100-125-05


Black Large M3: #FK0100-220-05

RevoFit2™


RevoFit2 Kit: (The next generation! Replaces both RevoFit Versa and Direct)
Smaller footprint, Longer lace feeder to fit larger sockets, For both inline and remote mounting, Available for both Lamination and Diagnostic (Optimize your RevoFit2 design on the Diagnostic Socket), Available in 10 packFootFit™

Available Kits

RevoFit2 Lamination Kit PK2000-320-05

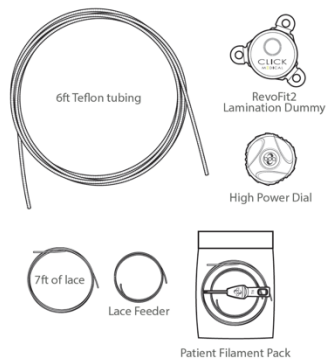


RevoFit2 Diagnostic Kit PK2150-320-05

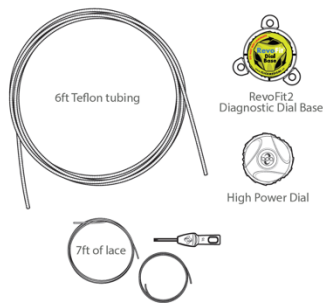


Improved Design



For Inline Mounting (Direct)

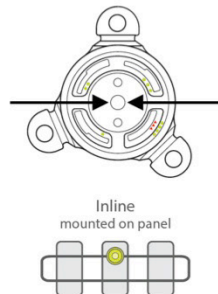

For Remote Mounting (Versa)

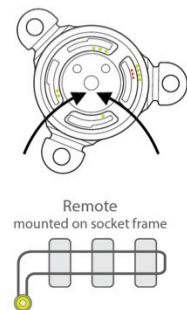

> RevoFit2™:

RevoFit2 Lamination Kit: #PK2000-320-05

RevoFit2 Diagnostic Kit: #PK2150-320-05



CLICK

1205 Hilltop Parkway, W101
Steamboat Springs, Co 80487
970.670.7012 or info@clickmedical.co