FOLLO CORPUS SX / SXT / SXG

FOLLO CORPUS SX / SXT / SXG

We congratulate you on choosing a Follo Corpus product. This instruction makes it able for you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.

Contents: Check list, unpacking/installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list

ORDER NO.:

COLOUR:

SERIAL NO.:

FUNCTION TEST:

☐ Bottom frame (screws, plastic parts, surface)	☐ Cushions (colour, screws, surface)
☐ Top frame (screws, plastic parts, surface)	☐ Paper roll holder
☐ Surfaces (no damages)	☐ Foot pedal, lowering/raising.
☐ Raising (function/noise)	☐ 110 Voltage ☐ 220 Voltage

ASSEMBLED (date/sign.):

END CONTROLLED (date/sign.):

PACKING CONTROL: (date/sign.):

FOLLO FUTURA AS - POSTBOKS 112 - 1430 Ås - NORWAY

WWW.FOLLO-FUTURA.NO

EC DECLARATION OF CONFORMITY FOR MACHINERY AND MEDICAL DEVICES

Manufacturer: Follo Futura AS

Address: Brekkeveien 43
1430 Ås
NORWAY

Herewith declares that: Follo Corpus SX (126-22xxxx)

is in conformity with the provisions of the Machinery Directive (Directive 2006/42/EF),
Medical Device Directive (93/42/EEC annex VII)

Class 1.

Ås, Norway 31.01.2011

Mette Veiby
Managing Director

EC DECLARATION OF CONFORMITY FOR MACHINERY AND MEDICAL DEVICES

Manufacturer: Follo Futura AS

Address: Brekkeveien 43
1430 Ås
NORWAY

Herewith declares that: Follo Corpus SXT (126-225200T)

is in conformity with the provisions of the Machinery Directive (Directive 2006/42/EF),
Medical Device Directive (93/42/EEC annex VII)

Ås, Norway

Mette Veiby
Managing Director

EC DECLARATION OF CONFORMITY FOR MACHINERY AND MEDICAL DEVICES

Manufacturer: Follo Futura AS

Address: Brekkeveien 43
1430 Ås
NORWAY

Herewith declares that: Follo Corpus SXG (126-225300G)

is in conformity with the provisions of the Machinery Directive (Directive 2006/42/EF),
Medical Device Directive (93/42/EEC annex VII)

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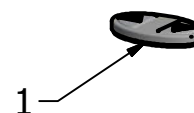
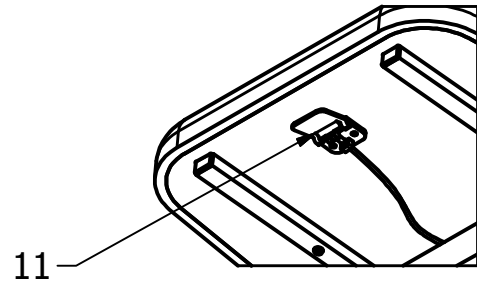
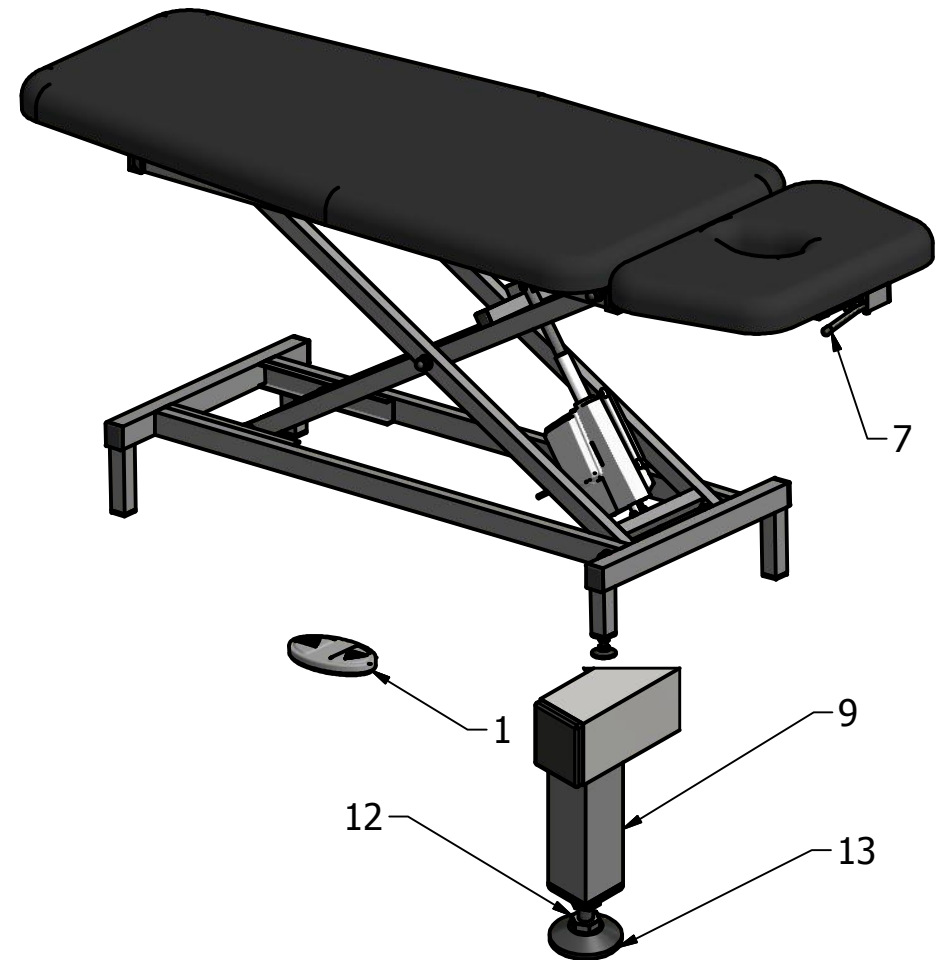
INTENDED USE, SX

Follo Corpus SX is a physiotherapy/examination/treatment table intended for treating patients.

INTENDED USE, SXT

Follo Corpus SXT is an examination and treatment table intended for treating patients.

INTENDED USE, SXG

Follo Corpus SXG is a gynaecology examination and treatment table intended for treating patients.

CHECK THE PRODUCT AS SOON AS IT IS DELIVERED

Inform the carrier immediately if you notice any damage to the product from transport. Keep the packaging until the carrier has checked the products and both agree to sign for the delivery.

Unpacking and installation

1. Be careful when opening the packing. Knives or sharp objects must NOT be used because these can damage the product.
2. Check to see whether the product was damaged during transport or not.
3. Test the table's functionalities.

Safety information

1. Familiarise yourself with thearm
2. table's handles and switches before you start to use it.
3. Make certain that nothing can be caught or pinched in the mechanism when making adjustments.
4. The table must not be subject to excess dynamic loads beyond those stated on the product specification sheet.
5. Make certain the table is standing on a flat and sturdy floor.
6. Make certain the wheels are locked before beginning treatment.

Guarantees

Follo Futura offers a 2-year guarantee on materials and manufacturing errors, on the condition the product is used in accordance with its intended use and operating instructions. Wear and damage to parts with surface treatments are not covered by the guarantee. Damage from abnormal use, or damage caused by exceeding load specifications, is not covered by the guarantee. The instructions for use and maintenance must be followed for the guarantee to be valid. The product has an anticipated lifetime of 5 years, assuming the product is subject to regular maintenance as described in these instructions. The product must not be modified in any way or used with other than the original parts. Producer liability for CE-labelling is revoked if any modifications are made to the product.

- The table has a 6-month guarantee on the cushions, in case of production errors. (This does not include damage during transport or cutting damage when opening the packing).
- The table has a 2-year guarantee on electrical parts.
- The table has a 5-year production guarantee on mechanical constructions.

This table has been mounted, checked and approved pursuant to the company's quality assurance system.

User's Manual, SX / SXT / SXG

Height adjustments:

Use the foot pedal (1) to adjust the table to the desired height.

Servicing the brake:

The wheel brake can be engaged from one side of the table. The product is not equipped with a main locking mechanism, but the brakes function in pairs; one pair at the head cushion and one pair at the foot of the table. The brake pedal (4) is marked with a red and a green pedal. Press down on the red pedal to engage the brake. **PLEASE NOTE!** Both brakes must be engaged to keep the table in a stable position. Press down on the green pedal to release the brake.

User's Manual, SX / SXG

Servicing the head cushion:

The head cushion (8) is adjusted using the handle located below the head cushion (7). Pull the handle toward you and adjust the cushion to the desired position, then release the handle. **PLEASE NOTE!** The head cushion should not be used as a chair.

User's Manual, SX

Servicing the back cushion:

The back cushion (10) is adjusted using the handle (11) located under the back cushion. Push the handle inward to set the cushion at the desired position, then release the handle to lock the cushion in place.

Adjust the frame:

The table's stability is adjusted by moving the adjustment leg (9). First loosen the backnut (12), then move the adjustable foot (13) until the table legs rest firmly on the floor, then lock the adjustment leg by tightening the backnut.

User's Manual, SXT

Servicing the head cushion:

The handle (3) in front/below the head cushion releases the lock on the head cushion so the cushion can be rotated up or down. The head cushion will remain in place when you release the handle. **PLEASE NOTE!** Be certain the handle (3) points downward so the head cushion (2) and handle are not jammed together.

User's Manual, SXG

Servicing the legs supports:

Loosen the wheel (6) and pull the pipe outward in a horizontal direction to the desired position, then tighten the wheel. To adjust the leg supports to the correct height: Loosen the handle (5), adjust to the desired height, then tighten the handle firmly.

Maintenance / regular inspections

Check the following items once every three months:

1. Damage or wear to wires.
2. Check to see that the height adjustment and all servicing handles and pedals are working properly.
3. Apply a thin layer of standard silicone spray on the glide surfaces.
4. Check the Seeger rings and screws to make certain these have not loosened or fallen off.
5. Check to see that the brake function is working properly.

Cleaning

The synthetic leather must be cleaned frequently to protect its lasting quality. A soft rag and warm soapy water should be used. Mild soap and soap flakes are recommended, remaining soap must be removed using cold water – then dry using a soft rag. Remove any stains using a soft brush.

The synthetic Skai leather is resistant to mild acids, alkali liquids and normal stains. Stains from ballpoint pens, lipstick, newspapers and foods can stick to the leather permanently, so these must be removed immediately using a moist soft rag or sponge with soapy water. Do not put hot or cold packs directly on the leather. Solvents, wax polish, strong synthetic detergents or chemical detergents must not be used. The use of these substances will destroy the plastic and it will harden and crack.

Recycling

The actuator, control box and foot pedal contacts should be delivered to an electrical waste container. The frame, scissor legs, top frame and cushion frame can be discarded in a metal recycling bin. The plastic products and cushions should be thrown into the general waste bin.

The electric motor

The electric motor is ready for use and has been tested at the factory. In the event of a malfunction try the following solutions:

1. Check the plug to see if it is sitting properly in the socket.
2. Check to see that the electrical contacts on the motor are properly fastened.

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